# ResCUBE™

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# ResCUBE™

Ligament System for Spine Stabilisation  
**Single use sterile product**

## **DESCRIPTION**

The ResCUBE device is an implant with the purpose of correction and stabilisation of the spine during fusion of the instrumented levels.

It consists of:

- A braid for correction and stabilisation of the levels, through tension;
- A malleable lamella for guidance of the braid during surgery and attached to the braid through a thermal-retractable sleeve. The malleable metal lamella and the sleeve linking it to the braid are not intended for long term implantation and must be removed after placement of the implant.
- A connector holding the tension on the braid

## **MATERIALS**

Braid	Polyethylene Terephthalate	For long term implantation
Connector	Titanium TA6V ELI	
Malleable lamella	Stainless Steel	Not for long term implantation
Shrink Sleeve	Polyolefin	

All used materials are non resorbable and not of human or animal origin.

## **INDICATIONS**

ResCUBE is indicated for the following surgical treatments:

- Spinal trauma surgery that can be treated with sublaminar, interspinous or facet ligamentoplasty.
- Spinal degenerative pathologies, as an adjunct to intervertebral fusion
- Spinal reconstructive surgery incorporated into constructions with the aim of correcting spinal deformities such as scoliosis, kyphosis and spondylolisthesis.

## **CONTRA-INDICATIONS**

The ResCUBE device should not be used in the following cases:

- General or local infection in the intervention area
- Local inflammation in the intervention area
- Pregnancy
- Immune deficiency
- Any pathology affecting bone metabolism, such as osteopenia, osteoporosis, metastasis, tumour... can compromise the expected mechanical support of the implant, and therefore, the success of the surgery
- Insufficient tissue or muscle coverage
- Suspected or confirmed sensitivity to at least one of the components
- Excess weight or obesity can lead to mechanical overload on the implant, compromising the success of the surgery
- Any mental state or activity level incompatible with the postoperative follow-up can compromise the success of the surgery
- Smoking patients are more likely to present a non-satisfactory bone fusion or non-fusion
- Any other contra-indication to instrumented surgery of the spine

The described contra-indications can be absolute or relative and must be evaluated by the surgeon prior to any implant decision.

## **POTENTIAL UNDESIRABLE SECONDARY EFFECTS**

In addition to the risks linked to instrumented spine surgery, the following effects can be observed in rare cases:

- Unusual sensations, discomfort or pain due to the presence of the implant
- Migration, deformation or breakage of one or several components of the implant that can cause neurological, mobility complications or pain
- Infection, allergic or inflammatory reactions due to the components of the implant, or to the presence of micro particles (metallosis)
- Halting the growth of instrumented levels
- Partial or total loss of correction
- Intra-operative or post-operative lesions, fissures or fractures, in case of insufficient bone density
- Pseudo arthritis
- Pulmonary complications
- Death

In case of undesirable secondary effects, a second surgery may be necessary, in order to remove or replace one or several implants.

## **PRECAUTIONS FOR USE**

### *General precautions*

ResCUBE devices must be implanted by specialist spinal surgeons, trained in the surgical technique. The information contained in this leaflet is necessary but not sufficient to master the surgical technique.

### **IMPORTANT: DO NOT RE-USE - DO NOT RESTERILISE**

ResCUBE implants are supplied sterile and for single use. They must not be resterilised or re-used (potential risks include but are not limited to: loss of product sterility, risk of infection, loss of effectiveness, relapse).

Implants must be handled and stored with great care, in a dry place, protected from sunlight and at room temperature.

### *Pre-operative precautions*

The implantation decision must be taken by the surgeon, after informed consent of the patient, considering the indications, contra-indications and potential undesirable side effects.

Before the surgery, it is recommended to check the compatibility of devices which will be associated (size, materials...).

### *Peroperative precautions*

Before opening, it is recommended to check the packaging is untampered with. Any damage on packaging may jeopardise the implant sterility. If the packaging is damaged, the implant must not be used, or resterilised.

Any implant whose expiration date has passed must not be used.

After opening, it is recommended to check the implant integrity.

Any implant with a damaged surface must not be implanted.

The placement of the implant must be done with the appropriate ancillaries, following the surgical technique recommended by COUSIN BIOTECH.

The malleable metal lamella and the sleeve linking it to the braid are not intended for long term implantation and must be removed after placement of the implant.

### *Post-operative precautions*

The patient must be informed of the post-operative care and of precautions.

## **RETRIEVAL AND DISPOSAL OF DEVICES**

Retrieval and handling should be done in compliance with ISO 12891-1:2011 « Implants for surgery – Retrieval and analysis of surgical implants » Part 1: « Retrieval and Handling ».

Any removed device must be sent back for analysis, following the current protocol. The latter is available on request to COUSIN BIOTECH. It is important to note that any implant that cannot be cleaned and disinfected before despatch must be contained in a sealed package.

The disposal of removed medical devices must be conducted according to national standards for the disposal of infectious hazards waste.

The disposal of a non-implanted device is not subject to specific recommendations.

## **INFORMATION REQUESTS AND CLAIMS**

Following its quality policy, COUSIN BIOTECH is committed to make every effort to produce and supply high quality medical devices. However, if a health professional (client, user, prescriber ...) has a complaint or cause for dissatisfaction with a product in terms of quality, safety or performance, he must inform COUSIN BIOTECH as soon as possible.

In case of dysfunction of an implant or if it contributed to causing serious adverse effects for the patient, the health centre must follow the legal procedures in its country, and inform COUSIN BIOTECH as soon as possible.

For any correspondence, please specify the reference, batch number, details of a reference and a comprehensive description of the incident or complaint.

Brochures, documentation and surgical techniques are available on request from COUSIN BIOTECH and its distributors.

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