BIOLOG[®]

LIGAMENTS ARTICULAIRES NON RESORBABLES STERILES

en	Instructions for use	Page	2
fr	Notice d'instructions	Page	3
de	Gebrauchsanweisung	Seite	4
it	Istruzioni per l'uso	Pagina	5
es	Instrucciones de uso	Pagina	6

COUSIN BIOTECH

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STERILE PRODUCT FOR SINGLE USE

DESCRIPTION

Reinforcing ligaments and tendons for orthopedic surgery

MATERIALS

Polyethylene terephthalate (No resorbable)

Stainless steel

Neither human nor animal origin.

INDICATIONS

The ligaments are intended for the surgical reinforcement of instabilities in different joints. They can be located intra- or extra-articularly. The reinforcing ligament or tendinous should always be used to complement a suture or natural tissue graft.

CONTRAINDICATIONS

Do not use:

- In a septic environment or in an infected site,
- allergy to one of components
- growing child.

POSSIBLE ADVERSE SIDE EFFECTS

- Routine surgical complications
 - -superficial or deep infection
 - -anesthesia complications
 - local hematoma, blood loss
- ·Postoperative loss of motion, frozen joint
- ·Inflammatory / allergic reactions / arthro fibrosis (around 1%) / Bone resorption (osteolysis)
- ·Ongoing instability episodes
- ·Neurovascular injury, peripheral neuropraxia
- ·Wound dehiscence, serous leak
- ·Post-surgical pain, persistent pain
- ·Ligament injury
- ·Permanent elongation of the ligament / Rerupture of the ligament

PRECAUTIONS FOR USE

Inspect the packaging to be sure it is intact (do not use if the labels and/or the protective packaging are damaged).

Do not use if the device is damaged.

DO NOT REUSE

DO NOT RESTERILIZE

COUSIN BIOTECH does not make any guarantees or recommendations as regards the use of a particular brand of fixation devices. The properties of these devices are the subject of modifications by the manufacturer and over which COUSIN BIOTECH has no control. Suture clips or sharp mounting screws must not be used.

This device should be implanted **only** by a qualified surgeon who has been trained in the use of this product.

Postoperative precautions

The device has not been tested for heating or migration in the MR environment.

The device has been evaluated for safety and compatibility in the MR (Magnetic Resonance) environment. According to medical literature and in view of the materials used in the implant, the device can be considered as MR safe.

PRECAUTIONS FOR STORAGE

To be stored in a dry place, protected from sunlight, at room temperature in the original packaging.

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