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SUMMARY OF SAFETY AND CLINICAL PERFORMANCE OF IntraSPINE (Sterile Devices for Interspinous Space with Laminar Support)

en	<u>SUMMARY OF SAFETY AND CLINICAL PERFORMANCE OF IntraSPINE</u>
de	<u>KURZBERICHT ÜBER SICHERHEIT UND KLINISCHE LEISTUNG VON IntraSPINE®</u>
it	<u>SINTESI RELATIVA ALLA SICUREZZA E ALLA PRESTAZIONE CLINICA DI IntraSPINE®</u>
es	<u>RESUMEN DE SEGURIDAD Y RENDIMIENTO CLÍNICO DE IntraSPINE®.</u>
nl	<u>SAMENVATTING VAN DE VEILIGHEID EN KLINISCHE PRESTATIES VAN IntraSPINE®</u>

Table of Contents

INTRODUCTION	14
SECTION 1: SSCP FOR USERS/HEALTHCARE PROFESSIONALS	15
1. DEVICE IDENTIFICATION AND GENERAL INFORMATION	15
1.1. DEVICE TRADE NAME(S).....	15
1.2. MANUFACTURER'S NAME AND ADDRESS	15
1.3. MANUFACTURER'S SINGLE REGISTRATION NUMBER (SRN).....	15
1.4. BASIC UDI-DI	15
1.5. MEDICAL DEVICE NOMENCLATURE DESCRIPTION / TEXT	15
1.6. CLASS OF DEVICE	15
1.7. YEAR WHEN THE FIRST CERTIFICATE (CE) WAS ISSUED COVERING THE DEVICE.....	16
1.8. AUTHORISED REPRESENTATIVE IF APPLICABLE, NAME AND SRN	16
1.9. NOTIFIED BODY'S NAME AND SINGLE IDENTIFICATION NUMBER	16
2. INTENDED USE OF THE DEVICE.....	16
2.1. INTENDED PURPOSE	16
2.2. INDICATION(S) AND TARGET POPULATION(S)	16
2.3. CONTRAINDICATIONS AND/OR LIMITATIONS	16
3. DEVICE DESCRIPTION.....	17
3.1. DESCRIPTION OF THE DEVICE	17
3.1.1. GENERAL DESCRIPTION	17
3.1.2. LIST OF REFERENCES.....	18
3.1.3. PRINCIPLES OF OPERATION	19
3.1.4. KEY FUNCTIONAL ELEMENTS	19
3.1.5. MATERIALS AND CONTAINED SUBSTANCES	20
3.2. A REFERENCE TO PREVIOUS GENERATION(S) OR VARIANT(S) IF SUCH EXIST, AND A DESCRIPTION OF THE DIFFERENCES.....	20
3.3. DESCRIPTION OF ANY ACCESSORIES WHICH ARE INTENDED TO BE USED IN COMBINATION WITH THE DEVICE	20
3.4. DESCRIPTION OF ANY OTHER DEVICES and products WHICH ARE INTENDED TO BE USED IN COMBINATION WITH THE DEVICE	21
4. RISKS AND WARNINGS.....	22
4.1. RESIDUAL RISKS AND UNDESIRABLE EFFECTS	22
4.2. WARNINGS AND PRECAUTIONS	24
4.3. OTHER ASPECTS OF SAFETY INCLUDING A SUMMARY OF ANY FIELD SAFETY CORRECTIVE ACTION (FSCA INCLUDING FSN) IF APPLICABLE	26

5. SUMMARY OF CLINICAL EVALUATION AND INFORMATION ON POST-MARKET CLINICAL FOLLOW-UP	26
5.1. SUMMARY OF OTHER CLINICAL DATA RELATED TO EQUIVALENT DEVICE, IF APPLICABLE	26
5.2. SUMMARY OF CLINICAL DATA FROM CONDUCTED INVESTIGATIONS OF THE DEVICE BEFORE THE CE-MARKING, IF APPLICABLE	26
5.3. SUMMARY OF CLINICAL DATA FROM OTHER SOURCES, IF APPLICABLE	26
5.3.1. PMS DATA	36
5.4. AN OVERALL SUMMARY OF CLINICAL PERFORMANCE AND SAFETY	37
5.5. ONGOING OR PLANNED POST MARKET CLINICAL FOLLOW-UP	39
6. POSSIBLE DIAGNOSTIC OR THERAPEUTIC ALTERNATIVES	40
7. SUGGESTED PROFILE AND TRAINING FOR USERS	42
8. REFERENCE TO ANY HARMONISED STANDARDS AND COMMON SPECIFICATIONS APPLIED	42
BIBLIOGRAPHY	46
SECTION 2: SSCP FOR PATIENTS	48
1. DEVICE IDENTIFICATION AND GENERAL INFORMATION	48
1.1. DEVICE TRADE NAME(S)	48
1.2. MANUFACTURER'S NAME AND ADDRESS	48
1.3. BASIC UDI-DI	48
1.4. YEAR WHEN THE FIRST CERTIFICATE WAS ISSUED COVERING THE DEVICE	48
2. INTENDED USE OF THE DEVICE	48
2.1. INTENDED PURPOSE	48
2.2. INDICATIONS AND INTENDED GROUPS	49
2.3. CONTRAINDICATIONS	49
3. DEVICE DESCRIPTION	49
3.1. DESCRIPTION OF THE DEVICE AND MATERIALS / SUBSTANCES IN CONTACT WITH PATIENT TISSUES	49
3.1.1. GENERAL DESCRIPTION	49
3.1.2. LIST OF REFERENCES	51
3.1.1. MATERIALS AND CONTAINED SUBSTANCES	51
3.2. INFORMATION ABOUT MEDICINAL SUBSTANCES IN THE DEVICE, IF ANY	52
3.3. DESCRIPTION OF HOW THE DEVICE IS ACHIEVING ITS INTENDED MODE OF ACTION	52
3.4. DESCRIPTION OF ACCESSORIES, IF ANY	53
4. RISKS AND WARNINGS	53
4.1. HOW POTENTIAL RISK HAVE BEEN CONTROLLED OR MANAGED	53
4.2. REMAINING RISKS AND UNDESIRABLE EFFECTS	53
4.3. WARNINGS AND PRECAUTIONS	54

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INTRODUCTION

This Summary of Safety and Clinical Performance (SSCP) is produced in alignment with the requirements of COUSIN BIOTECH's Clinical Evaluation Plan (TF02_CER_ISSUE-2) and REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL, referencing the MDCG 2019 – 9 - Summary of safety and clinical performance: A guide for manufacturers and notified bodies.

This SSCP is intended to provide public access to an updated summary of the main aspects of the safety and clinical performance of the device. The SSCP is not intended to replace the Instructions For Use (IFU) as the main document to ensure the safe use of the device, nor is it intended to provide diagnostic or therapeutic suggestions to intended users or patients.

This SSCP is an important source of information for intended users – both healthcare professionals and if relevant for patients. It is one of several means intended to fulfil the objectives of the Medical Device Regulation (MDR) to enhance transparency and provide adequate access to information. The SSCP for Sterile Devices for Interspinous Space with Laminar Support composed of IntraSPINE includes both information intended for healthcare professionals (Section A) as well as for the patient (Section B).

The SSCP for IntraSPINE is based on the following documentation pertaining to the IntraSPINE:

- Clinical Evaluation Report (CER)
- Post-Marketing Surveillance (PMS) plan and regular reports
- Post-Marketing Clinical Follow-up (PMCF) plan and regular reports
- Design verification/validation reports
- If appropriate, the IFU

The SSCP shall be reviewed on an annual basis and updated if needed to ensure that any clinical and/or safety information in the SSCP remains correct and complete after implementation of the PMCF reports and of the Periodic Safety Update Report (PSUR). Any pertinent modification in the documentation pertaining to IntraSPINE shall be implemented in all sections of the SSCP if needed.

The SSCP of IntraSPINE must contain all the information detailed in this template. However, the manufacturer may add further information from the Technical Documentation (TD) of the device to enhance the comprehension of the mandatory information if it does not affect the readability of the SSCP and if it excludes any element of a promotional nature.

For the sake of objectivity and impartiality to the patients, information in the SSCP shall adequately summarize both favorable and unfavorable data.

The IFU shall provide all relevant information needed to directly access the SSCP in Eudamed. The IFU shall be updated regularly on this database. The following applies to the IFU:

- It shall state that the SSCP is available in the European database on medical devices (Eudamed), where it is linked to the Basic UDI-DI.
- It should provide the URL to the Eudamed public website: <https://ec.europa.eu/tools/eudamed>
- It should state the value of the Basic UDI-DI. Alternatively, another metadata can be stated provided it can be used to unambiguously search and find the intended SSCP in Eudamed.

The MDR requires that the SSCP shall be written in a way that is clear to the intended user and, if relevant, to the patient. The SSCP should therefore be translated into the languages accepted in the Member States where the device is sold/envisaged to be sold. By analogy, this is also a requirement for an IFU.

Manufacturer's reference number for SSCP: TF(02)_SSCP_INTRASPINE

SECTION 1: SSCP FOR USERS/HEALTHCARE PROFESSIONALS

This Summary of Safety and Clinical Performance (SSCP) is intended to provide public access to an updated summary of the main aspects of the safety and clinical performance of the device.

The SSCP is not intended to replace the Instructions For Use as the main document to ensure the safe use of the device, nor is it intended to provide diagnostic or therapeutic suggestions to intended users or patients.

The following information is intended for users/healthcare professionals. A summary intended for patients can be found in Section 2.

1. DEVICE IDENTIFICATION AND GENERAL INFORMATION

1.1. DEVICE TRADE NAME(S)

The family identified and concerned by the SCCP is Sterile devices for interspinous space with laminar support, included: IntraSPINE®.

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1.3. MANUFACTURER'S SINGLE REGISTRATION NUMBER (SRN)

The Single Registration Number (SRN) of the manufacturer is FR-MF-000001179.

1.4. BASIC UDI-DI

376018557INTRASPINEIMP47

1.5. MEDICAL DEVICE NOMENCLATURE DESCRIPTION / TEXT

The EMDN code of the Sterile Devices for Interspinous Space with Laminar Support is: P09070305, SPINAL STABILISERS, DYNAMIC TYPE

The GMDN code of the Sterile Devices for Interspinous Space with Laminar Support is: 58446, SPINAL DYNAMIC STABILIZATION SYSTEM

1.6. CLASS OF DEVICE

All the Sterile Devices for Interspinous Space with Laminar Support included in this EC technical file are in Class III according to the Annex VIII, chapitre III, point 5,4, Rule 8 dash 9 of the classification criteria of the Regulation (EU) No 2017/745.

1.7. YEAR WHEN THE FIRST CERTIFICATE (CE) WAS ISSUED COVERING THE DEVICE

The CE marking certificate number of IntraSPINE® is FR19/81843462 under Directive 93/42/CEE. The device INTRASPINE® is CE marked since March 2007 (93/42/EEC).

The CE certificate was obtained for the RCBINTRAXX and BIOINTRAXX references, in accordance with European Regulation 2017/745, on 2025-05-09. The certificate numbers are as follows:

- EU Quality Management System Certificate MDR 00019-A
- EU Technical Documentation Assessment Certificate MDR 00019-B

Note: The references used under the Directive 93/42/EEC have been added in order to allow distributors familiar with the former references to continue using these historical references. There are no differences between references INTRACALXX and RCBINTRAXX. They are strictly identical concerning manufacturing. The RCBINTRAXX references previously used under the MDR have been removed in order to streamline reference management within the company's system.

1.8. AUTHORISED REPRESENTATIVE IF APPLICABLE, NAME AND SRN

Not applicable.

1.9. NOTIFIED BODY'S NAME AND SINGLE IDENTIFICATION NUMBER

The Notified Body (NB) for the IntraSPINE® and the validation of this SSCP is KIWA. Its Single identification number is CE 0476.

2. INTENDED USE OF THE DEVICE

2.1. INTENDED PURPOSE

IntraSPINE® is intended to be used as a dynamic interspinous lumbar spacer with interlaminar support. This device belongs to the non-fusion spinal products offering an alternative to arthrodesis surgery.

2.2. INDICATION(S) AND TARGET POPULATION(S)

The IntraSPINE® is an interspinous with laminar support prosthesis for discal or facet assistance. This device is indicated for use from L1-S1 in skeletally mature patients with at least moderate impairment in function and who experience relief from symptoms of leg/buttocks/groin pain in flexion, this being either associated or not with back pain, and who have undergone at least 6 months of non-operative treatment. The indications of the IntraSPINE® prosthesis are as follows:

- Arthropathic facet-syndrome
- Foraminal stenosis
- Degenerative discopathy

2.3. CONTRAINDICATIONS AND/OR LIMITATIONS

The IntraSPINE® prosthesis is contraindicated under the following circumstances:

- Allergy to silicone or one of its other components
- Growing child
- Infected site
- Pregnant woman

The following factors could compromise successful implantation:

- Severe osteoporosis
- Major deformations of the spinal column
- Local bone tumours
- Systemic or metabolic disorders
- Infectious diseases
- Obesity
- Drug addictions
- Intense physical activity, including competitive sports or hard labour
- Surgery at more than two levels with IntraSPINE® prosthesis
- Spondylolisthesis from Grade 2 to 4
- Intervertebral disc with Pfirrmann classification Grade 5
- S1 spinous process length inferior to 18mm when implanted in L5/S1

3. DEVICE DESCRIPTION

3.1. DESCRIPTION OF THE DEVICE

3.1.1. GENERAL DESCRIPTION

The IntraSPINE® prosthesis is an interlaminar medical device that comprises a dimethyl siloxane (silicone) wedge (available in several sizes¹) adapted to the intervertebral space and one or two ligaments in polyethylene terephthalate.

The wedge is covered with woven polyethylene terephthalate with an area covered by sheeting on the front face in the medullary zone.

Tension is applied to the ligament with a titanium loop (TA6V) or with a crimping system in titanium (T40) (depending on the choice of ligament used).

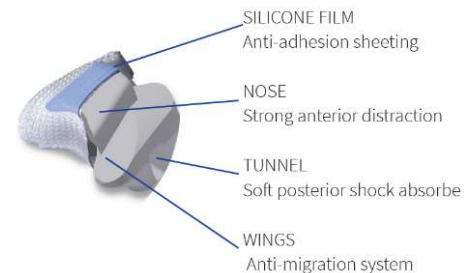
Figure 1 - IntraSPINE® wedge

¹ The wedge is available in several sizes (8mm, 10mm, 12 mm, 14 mm and 16mm) to fit to patient 's anatomy

Figure 2 - IntraSPINE® device (BIOINTRAXX variant)

The IntraSPINE® is an elastic support device with a cushioning effect. Its interlaminar implantation enables to be closed to the centre of rotation, thereby producing the cushioning effect and providing tension on the inter-spinous ligamentary structures due to the elasticity of the wedge.

The principal function of the ligament surrounding the upper and lower vertebrae in relation to the area receiving the prosthesis is to assist the posterior ligament during flexion.

The second ligament stabilizes the interlaminar device on its site.

The intended use of the medical device IntraSPINE® is to treat the Degenerative (intervertebral) disc disease (DDD) and its consequences.

IntraSPINE® is an invasive and implantable sterile device, sterilized by gamma radiation for single use.

3.1.2. LIST OF REFERENCES

Hereafter, the references, and their description:

PRODUCT CODE	PRODUCT SHAPE	DESCRIPTION
INTRACAL08		Wedge 8mm covered by a knitting +1 ligament with a titanium loop
INTRACAL10		Wedge 10mm covered by a knitting +1 ligament with a titanium loop
INTRACAL12		Wedge 12mm covered by a knitting +1 ligament with a titanium loop
INTRACAL14		Wedge 14mm covered by a knitting +1 ligament with a titanium loop
INTRACAL16		Wedge 16mm covered by a knitting +1 ligament with a titanium loop
BIOINTRA08		Wedge 8mm covered by a knitting +1 ligament with a titanium loop + 1 needled ligament with a hardened loop + crimp device on nylon wire

BIOINTRA10		Wedge 10mm covered by a knitting +1 ligament with a titanium loop + 1 needed ligament with a hardened loop + crimp device on nylon wire
BIOINTRA12		Wedge 12mm covered by a knitting +1 ligament with a titanium loop + 1 needed ligament with a hardened loop + crimp device on nylon wire
BIOINTRA14		Wedge 14mm covered by a knitting +1 ligament with a titanium loop + 1 needed ligament with a hardened loop + crimp device on nylon wire
BIOINTRA16		Wedge 16mm covered by a knitting +1 ligament with a titanium loop + 1 needed ligament with a hardened loop + crimp device on nylon wire

Table 1 References of Sterile Devices for Interspinous Space with Laminar Support

3.1.3. PRINCIPLES OF OPERATION

IntraSPINE® has been designed to be implanted in the interlaminar area.

From its implantation in the interlaminar space, the device is closer of the AIR (Axis instantaneous of rotation), and this allows maintain a better range of motion.

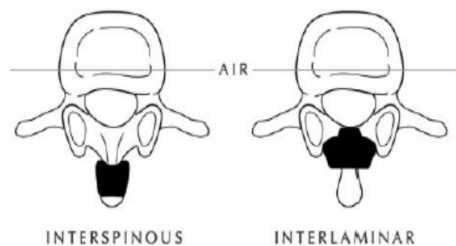
Figure 3 - Interspinous versus interlaminar implantation

This design allows three functionalities:

- The nose (anterior part) is rigid and positioned between laminae in order to recover a normal disc height.
- The tunnel in the posterior part allows the cushioning effect between the spinous processes.
- The wings (posterior part) represent the security of the device because they are developed to avoid any anterior migration.

Specific dimensions have been established during the design development of the prosthesis to perfectly match the anatomy of the lumbar segment and to ensure the claimed clinical performances.

During the surgery, a distraction is made by moving the 2 spinous processes with a specific ancillary. This helps the insertion of IntraSPINE® implant in the interlaminar area. The nose of the prosthesis is placed against two laminae and the body containing the tunnel is between upper and lower spinous process. Then, the ligament surrounding the upper and lower vertebrae maintains the spinous process in their natural position and consequently maintains the compression on the nose. The ligament brings additional safety against potential posterior migration.

3.1.4. KEY FUNCTIONAL ELEMENTS

IntraSPINE® is composed of the following key functional elements:

- A wedge, covered by a PET knitting with a silicone sheeting,
- A ligament with a titanium buckle,
- A second needle ligament with hardened loop and a crimping system (only for BIOINTRAXX references).

See the figure 2 for the illustration of key functional elements of IntraSPINE®.

3.1.5. MATERIALS AND CONTAINED SUBSTANCES

Duration of use or contact with the body

Long term implantation - Permanent Implant > 30 days

Materials or substances in contact with the patient tissues

The body part on which the device acts or with which it interacts is lumbar vertebrae.

BRAND NAME	PART OF THE DEVICE	COMPONENTS	RAW MATERIALS (Commercial name)
IntraSPINE®	Wedge	Nude Wedge	Dimethyl siloxane MED4765
		Glue	Dimethyl siloxane MED20000
		Sheeting	Dimethyl siloxane MED4750
		Sewing thread	Polyethylene terephthalate
		Knitting	Polyethylene terephthalate
	Ligaments	Braid	Polyethylene terephthalate
		Sewing thread	Polyethylene terephthalate
		Loop	Titanium TA6V

Table 2 Materials or substances in contact with the patient tissues for IntraSPINE®

Other substances

A medicinal substance (including a human blood or plasma derivative)	No
Tissue(s) or cells of human or animal origin, or their derivatives	No
Substances or combinations of substances that are absorbed by or locally dispersed in the human body	No
Materials incorporated into the device that contain or consist of CMR (carcinogenic, mutagenic or toxic to reproduction) substances or endocrine-disrupting substances	No
Materials that could result in sensitisation or an allergic reaction by the patient or user	No

3.2. A REFERENCE TO PREVIOUS GENERATION(S) OR VARIANT(S) IF SUCH EXIST, AND A DESCRIPTION OF THE DIFFERENCES

Not applicable, as there are no previous versions of the device.

3.3. DESCRIPTION OF ANY ACCESSORIES WHICH ARE INTENDED TO BE USED IN COMBINATION WITH THE DEVICE

Special REUSEABLE instruments have to be used to implant this device: 1 large bone forceps, distraction forceps, 5 trial devices (5 sizes), 1 left and 1 right wire guides, 1 implant holder, 1 impactor, 1 ligament tightener and 1 crimp tool (only for BIOINTRAXX).

They are sold as a complete set: reference KITACINTRA for INTRACALXX references or KITBIOINTR for BIOINTRAXX references.



PRODUCT CODE	PICTURE	DESCRIPTION
FANINTRA08 FANINTRA10 FANINTRA12 FANINTRA14 FANINTRA16		Reusable and removable phantom in stainless steel for IntraSPINE® products, sizes 08, 10, 12, 14 and 16 – In stainless steel 316L
PRTFAINTRA		Trial device holder
PCALEINTRA		Implant holder
ROGNINTRAL		Large bone rongeur forceps
IMPACINTRA		Impactor
RCBANTENDU		Ligament tightener
FBIOMC2003		Crimping tool
ANCINTRALF		Left wire guide
ANCINTRARG		Right wire guide
PINCEINTRA		Distraction forceps

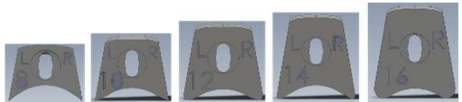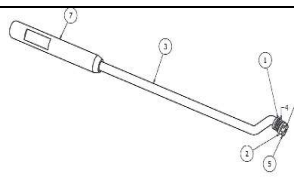Table 3 Accessories not included but necessary to use with IntraSPINE®

3.4. DESCRIPTION OF ANY OTHER DEVICES AND PRODUCTS WHICH ARE INTENDED TO BE USED IN COMBINATION WITH THE DEVICE

Not applicable.

4. RISKS AND WARNINGS

4.1. RESIDUAL RISKS AND UNDESIRABLE EFFECTS

Per instructions for use NOT293 (INTRACALXX) and NOT310 (BIOINTRAXX), complications associated with using the device under evaluation are:

Complications likely related to the device

- Infection of the prosthesis
- Device breakage (rupture of the ligament, fixed wing breakage, or knitting tearing)
- Uncrimping of the needle
- Device loosening permanent elongation of the ligament
- Device migration displacement of the wedge
- Spondylolisthesis to the dynamically stabilised segment
- Persistent or worsening pain attributed to the dynamically stabilised level, requiring revision surgery with interbody fusion at that level
- Spinous process fractures, either intraoperatively, post-operatively, or upon device removal (removal due to infection), which can be associated with implant dislocation
- Late reoperation due to recovery failure
- Disability
- Discomfort
- Wrong level implantation, with inappropriate or inaccurate positioning (posterior or wrong level)
- Seroma/swelling around the device
- Inflammatory reaction, allergy, or foreign body reaction
- In certain cases, the medical instrument had to be removed, with pedicle screw fusion performed.

Known complications of the procedure (if applicable)

- Pseudo-meningocele, fistula, dural tear, persistent CSF leakage, and meningitis
- Loss of neurological function
- Cauda equina syndrome, neuropathy, neurological deficits, arachnoiditis, muscle loss, compression around nerves, and pain
- Urinary retention, loss of bladder control, or other urological system compromises
- Fracture, bone loss, or decreased bone mineral density
- Herniated nucleus pulposus and disc disruption or degeneration either at, above, or below the surgery level, with adjacent segment degeneration
- Non-union or pseudo-arthritis, delayed union, or mal-union
- Cessation of any potential growth of the operated spine portion
- Either loss of or increase in spinal mobility or function
- Haemorrhage, haematoma, oedema, embolism, stroke, excessive bleeding, phlebitis, wound necrosis, wound dehiscence, wound infection, damage to blood vessels, or other cardiovascular system compromises
- Development of respiratory problems

Hereafter, the assessment of the risk associated with the use of IntraSPINE[®], based on the clinical data.

⁽¹⁾ Data from the state of the art

⁽²⁾ Data from Post Market Clinical Follow-up studies performed by Cousin Biotech (See the section 5.3.2 PMCF DATA) and scientific literature (See the section 5.3.1 Literature review).

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Undesirable side effect	Source	Frequency		Magnitude	Duration	Mitigation
		Expected ⁽¹⁾	Observed ⁽²⁾			
Device loosening	Loss of mechanical properties	1.9%	None	Variable	Until revision surgery	Choice of specific long-term materials
Device breakage	Loss of mechanical properties	1.4%	Rare < 0,5%	Moderate	Until revision surgery	Loss of mechanical properties
Device migration	Wrong dimension of the product	3.7%	Up to 4.51% (without clinical symptom)	variable	Until treated if clinical symptoms associated	Several size available Ligament for fixation Surgical technique
Spinous process fracture	Surgical error	5.1%	None	Moderate	No treatment	IFU, surgeon training, Surgical brochure, specific instrument set for implantation
Infection	Contamination of the surgical field or the device	6.66%	4.34%	Severe	Until treated	Aseptic technic, precautions when using in contaminated environment
Pain worsening	Multifactorial including surgery, patient, devices	Up to 40%	Up to 40%	Very variable, can be related to disease recurrence, can lead to surgery revision	Variable	Mini invasive surgery
Wound problem (infection, dehiscence)	Multifactorial including surgery and patient	14%	1.77%	Variable, can be severe	Until treated if severe	Aseptic technic, precautions when using in contaminated environment
Hematoma, Seroma	Multifactorial including surgery and patient	2% severe Minor Not described, frequent	Severe rare All including minor up to 11.3%	Variable, can be severe	Spontaneous resolutions or until treated if severe	Non-reducible
Allergy/Foreign body reaction/inflammatory reaction	Patient factor	Rare	Rare	Variable, can be severe	Until treated if severe	Biocompatible raw materials
Wrong level implantation	Surgical error	Rare	None	Variable	Until surgery revision	Surgical technic
Spine instability	Surgical act	Rare	None	Variable, can be severe	Until treated if severe	Use of posterior artificial

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Undesirable side effect	Source	Frequency		Magnitude	Duration	Mitigation
		Expected ⁽¹⁾	Observed ⁽²⁾			
						ligament / mini invasive surgery
Neurological or sensitive deficit	Multifactorial including surgery and patient	Up to 26%	None	Variable, can be severe	Until treated if severe	Surgeon trainings / IFU/ surgical technique
CSF leakage	Surgical technique	Observed in spine surgery 1.5%	5.93%	Variable can be severe	Until treated	Non-reducible – related to surgical technique
Revision for decompression	Surgery failure, patient factors	13%	Up to 13%	Moderate	Until treated	Patient selection – surgical technique
Conversion to fusion	Surgery failure, patient factors	10%	8.68%	Moderate	Until treated	Patient selection – surgical technique
Cardiovascular problem	Patient factor	Rare	None	Can be severe	Until treated if severe	Non-reducible – related to surgical act
Needle detachment	Surgery failure	Rare	None	Low	No treatment required	Surgical technique
Intraoperative complications	Multifactorial including surgery, patient, devices	1%	1%	Low	Variable	Surgical technique
Other non specific related to surgical act (patient discomfort, other non specific complications)	Multifactorial, including patient factors	Not quantified	Isolated cases	Low	Not quantified	Non-reducible – related to surgical act

Table 4 Assessment of the risk associated with the use of IntraSPINE®,

The data available is of sufficient amount and quality for the detection of undesirable side-effects and their frequency.

No unexpected undesirable events have been detected in the clinical data available (PMCF study and scientific literature).

All events detected are within expected limits and consistent with state of the art.

4.2. WARNINGS AND PRECAUTIONS

MRI SAFETY

The implants are composed of non-ferromagnetic materials and present a geometry not likely to generate induced current. Moreover, as they are fixed to tissues, they are unlikely to be mobilised. At first sight they

can be considered compatible with an MRI scan. Their safety, in particular in terms of heating and migration of the implant, has been evaluated through bibliographic data by comparison with data available on devices with similar composition, shape and use. This evaluation concluded a conditional compatibility for MRI scan up to 1.5 Tesla.

As a precautionary measure, it is recommended MRI scans be avoided within 48 hours of the implant placement, and to inform the person in charge of the scan of the recent implant placement, if such examination is essential.

It should be noted that devices presenting a high contrast with the biological environment can generate "artifacts" that must be taken into account for the perfect execution and interpretation of imaging exams.

For this purpose, the patient who has this implant must be advised to warn the health professionals concerned (radiologists and radiology operators) in as far as possible about the presence of this implant before these exams.

PRECAUTIONS FOR USE

Before usage, verify the integrity of the prosthesis, trial prostheses, ancillaries and packaging.

Do not use in the event of deterioration of the prosthesis and/or the labels, and/or the trial prostheses and/or the ancillaries and/or the packaging.

Do not use if the prosthesis has exceeded its use-by date.

Unpacking and manipulation of the implant must be carried out following asepsis standards in order to guarantee the sterility.

When opening the carton box, the surgeon has to make sure that the lot number, the model and the size of the prosthesis are equal to the data on the label of the inner pouch. Otherwise the use of the prosthesis is strictly prohibited.

Prevent the prosthesis, the trial prostheses and ancillaries from coming in to contact with objects that may alter their surface.

Operation conditions: Temperature from +32°C up to 42°C. Implants should be resistant to biologic fluids and the discharge of body tissues, which they contact during use. Therefore, the IntraSPINE® prosthesis has been validated for a long term implantation and, currently, Cousin Biotech as more than 10 years of feedback concerning these products.

The IntraSPINE® prosthesis, trial prostheses and ancillaries must only be used by a qualified surgeon who is trained in spinal surgery and to the surgical technique of IntraSPINE®. The information contained in this notice is necessary but not sufficient to control the surgical technique.

The IntraSPINE® is made with soft materials; any contact with harder ones may alter its mechanical behaviour and lifetime. It is mandatory to avoid contact with other materials.

The prescription of the device is decided on by the surgeon, the only person qualified to do so.

Topping off is not contra-indicated but special attention is required. IntraSPINE® is made with soft materials; any contact with harder ones may alter its mechanical behaviour and lifetime. It is mandatory to avoid contact with other materials.

Use of the system of COUSIN BIOTECH ancillaries and specific trial prostheses is required for implantation of the IntraSPINE® prosthesis.

Implants must be handled and stored with great care, in a dry place, protected from sunlight and at room temperature.

Transportation: Transportation of the devices should be done in the closed vehicles in accordance with the rules for those transport vehicles. There are no special transportation requirements for the device.

IMPORTANT: DO NOT REUSE AND DO NOT RESTERILISE THE INTRASPINE® PROSTHESIS

As indicated on the product labelling, the prosthesis is designed for single use. It should in no event be reused and/or resterilized (potential risks include but are not limited to: loss of product sterility, risk of infection, loss of effectiveness, relapse).

POST-OPERATIVE PRECAUTIONS

The patient should be informed of the post-operative care and precautions to be observed.
The use of a brace may be necessary but is not mandatory. This decision can only be made by the surgeon.

STORAGE PRECAUTIONS

Keep in a dry place away from sunlight, at room temperature, in its original packaging.
Ancillaries and trial prostheses must be stored in their dedicated container or equivalent packaging to prevent any deterioration.

INSTRUCTIONS TO BE GIVEN TO THE PATIENTS BY THE SURGEON

The surgeon should inform the patient of the potential physical and psychological restrictions and consequences of implanting the device. The patient must be informed of the surgical risks and possible side effects. The surgeon should invite the patient to return for further consultation if the patient presents symptoms that appear abnormal. An implant card for the patient is provided by the healthcare professional at the clinic. This implant card provides information, for the patient, to identify the device and traceability elements as well as the name, address and website of the manufacturer. The surgeon shall invite the patient to scan the implant card immediately upon receipt to keep track of it in case of loss. The surgeon shall also inform the patient that the summary of the safety and performance characteristics of the device and the instructions for use can be found on the COUSIN BIOTECH website.

The instruction for use is in electronic format and can be downloaded from the COUSIN BIOTECH website (www.cousin-biotech.com/ifu).

4.3. OTHER ASPECTS OF SAFETY INCLUDING A SUMMARY OF ANY FIELD SAFETY CORRECTIVE ACTION (FSCA INCLUDING FSN) IF APPLICABLE

There has been no Field Safety Corrective Action (FSCA) or Field Safety Notice (FSN) since the CE marked in 2007.

5. SUMMARY OF CLINICAL EVALUATION AND INFORMATION ON POST-MARKET CLINICAL FOLLOW-UP

5.1. SUMMARY OF OTHER CLINICAL DATA RELATED TO EQUIVALENT DEVICE, IF APPLICABLE

Not applicable.

5.2. SUMMARY OF CLINICAL DATA FROM CONDUCTED INVESTIGATIONS OF THE DEVICE BEFORE THE CE-MARKING, IF APPLICABLE

Not applicable.

5.3. SUMMARY OF CLINICAL DATA FROM OTHER SOURCES, IF APPLICABLE

LITERATURE REVIEW

A systematic review of literature was conducted on MEDLINE, Cochrane library/ClinicalTrials.gov and Google Scholar. The following articles were identified and included in the clinical evaluation of IntraSPINE®.

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The following clinical situations are represented:

- Arthropathic (DDD) facet syndrome
- Disk Hernia / degenerative discopathy
- Soft stenosis

Performance outcome		Clinical data *		
		DDD/ 1 Arthropathic facet syndrome	DDD/2 Disk Hernia	Spinal stenosis
General performance outcome	Treat chronic low back pain	(Guizzardi 2011) ¹ (Guizzardi and Morichi 2015) ² (Gelder et al. 2020) ³ (Co-Minh 2021) ⁴ (Bae et al. 2017) ⁵	(Guizzardi 2011) ¹ (Corriero 2013) ⁶ (Galarza et al. 2014) ⁷ (Guizzardi and Morichi 2015) ² (Feng et al. 2020a) ⁸ (Co-Minh 2021) ⁴	(Guizzardi 2011) ¹ (Guizzardi and Morichi 2015) ² (Bistazzoni et al. 2017) ⁹ (Co-Minh 2021) ⁴
	Full Motion preservation	(Guizzardi and Morichi 2015) ² (Co-Minh 2021) ⁴	(Guizzardi and Morichi 2015) ² (Feng et al. 2020a) ⁸ (Co-Minh 2021) ⁴	(Guizzardi and Morichi 2015) ² (Bistazzoni et al. 2017) ⁹ (Co-Minh 2021) ⁴
Specific performance outcome	Elastic device with cushioning effect	(Guizzardi 2011) ¹ (Guizzardi and Morichi 2015) ² (Co-Minh 2021) ⁴	(Guizzardi 2011) ¹ (Guizzardi and Morichi 2015) ² (Feng et al. 2020a) ⁸ (Co-Minh 2021) ⁴	(Guizzardi 2011) ¹ (Guizzardi and Morichi 2015) ² (Bistazzoni et al. 2017) ⁹ (Co-Minh 2021) ⁴
	Ligaments assist the posterior ligament during flexion	(Guizzardi 2011) ¹ (Co-Minh 2021) ⁴	(Guizzardi 2011) ¹ (Co-Minh 2021) ⁴	(Guizzardi 2011) ¹ (Co-Minh 2021) ⁴
	The thin ligament stabilizes the interlaminar device on its site	Guizzardi 2011) ¹ (Guizzardi and Morichi 2015) ²	(Guizzardi 2011) ¹ (Feng et al. 2020a) ⁸	(Guizzardi 2011) ¹ (Bistazzoni et al. 2017) ⁹
	Respect of IAR of the segment	(Guizzardi 2011) ¹ (Guizzardi and Morichi 2015) ² (Co-Minh 2021) ⁴	(Guizzardi 2011) ¹ (Guizzardi and Morichi 2015) ² (Feng et al. 2020a) ⁸ (Co-Minh 2021) ⁴	(Guizzardi 2011) ¹ (Guizzardi and Morichi 2015) ² (Bistazzoni et al. 2017) ⁹ (Co-Minh 2021) ⁴
	Wings: anti migration system	(Guizzardi 2011) ¹ (Guizzardi and Morichi 2015) ²	(Guizzardi 2011) ¹ (Guizzardi and Morichi 2015) ² (Feng et al. 2020a) ⁸	(Guizzardi 2011) ¹ (Guizzardi and Morichi 2015) ² (Bistazzoni et al. 2017) ⁹
	Allows Minimal invasive surgery	(Guizzardi 2011) ¹ (Guizzardi and Morichi 2015) ² (Gelder et al. 2020) ³	(Guizzardi 2011) ¹ (Corriero 2013) ⁶ (Galarza et al. 2014) ⁷ (Guizzardi and Morichi 2015) ² (Feng et al. 2020a) ⁸	(Guizzardi 2011) ¹ (Guizzardi and Morichi 2015) ²
	Fast and simple procedure (distraction and optimal positioning of the device)	(Guizzardi 2011) ¹ (Guizzardi and Morichi 2015) ² (Gelder et al. 2020) ³ (Co-Minh 2021) ⁴	(Guizzardi 2011) ¹ (Corriero 2013) ⁶ (Galarza et al. 2014) ⁷ (Guizzardi and Morichi 2015) ²	(Guizzardi 2011) ¹ (Guizzardi and Morichi 2015) ² (Bistazzoni et al. 2017) ⁹ (Co-Minh 2021) ⁴

Performance outcome	Clinical data *		
	DDD/ 1 Arthropathic facet syndrome	DDD/2 Disk Hernia	Spinal stenosis
		(Feng et al. 2020a) ⁸ (Co-Minh 2021) ⁴	

Table 5 Summary of performances (based on clinical literature)

*** Clinical data**

Study reference	Type of study	Number of patients
(Co-Minh 2021)	Prospective, non comparative	231
(Guizzardi 2011)	Prospective non comparative	84
(Corriero 2013)	Retrospective case series	44
(Bae et al. 2017)	Prospective Case study	35
(Galarza et al. 2014) ^o	Prospective Comparative	22
(Guizzardi and Morichi 2015)	Retrospective	281 (duplicated in part with Guizzardi 2011 and Corriero 2013)
(Bistazzoni et al. 2017)	Prospective Case study	8
(Gelder et al. 2020)	non-randomized comparative retrospective analysis.	30
(Feng et al. 2020b)	retrospective analysis.	30

The intended benefits for the patient are the following:

- Positive impact on clinical outcome: Less pain and less disability
- Patient's quality of life: Improvement of the quality of life

According to the clinical data, there is a clear impact on the quality of life for the patients as illustrated by a decrease of pain and disability (Improvement of mean scores in all studies 73-93% patients).

Consequently, there is another benefit with public health impact as the percentage return to work is improved (Variable 20-60%).

Incident rates summary:

Incident typology	Rates reported	References
Migration without clinical symptom	4.51%	(Co-Minh 2021) ⁴
CSF leak	2.36%	(Co-Minh 2021) ⁴ (Galarza et al. 2014) ⁷
Infection of the prosthesis	Up to 4,34%	(Co-Minh 2021) ⁴
Problem with healing (swelling of the wound, aseptic collection)	1.77%	(Co-Minh 2021) ⁴ (Galarza et al. 2014) ⁷
Wound infection	Not reported with IntraSPINE	NA
Seroma/ fluid collection around the device with spontaneous resorption	5.39% -11,3%	(Guizzardi 2011) ¹ (Corriero 2013) ⁶
Hematoma	1.19%	(Co-Minh 2021) ⁴
Degeneration of adjacent level	4.36%	(Co-Minh 2021) ⁴
Conversion to fusion	8.28% 2.38% to 8.51%	(Co-Minh 2021) ⁴ (Guizzardi 2011) ¹ (Corriero 2013) ⁶
Revision for recurrence	0.59% 1.19% to 4.76%	(Co-Minh 2021) ⁴ (Guizzardi 2011) ¹ (Corriero 2013) ⁶
Hyper sensitivity reaction	Not reported with IntraSPINE	NA
Disc height loss (no clinical impact/no revision)	0.64%	(Co-Minh 2021) ⁴

Incident typology	Rates reported	References
Inaccurate positioning	7.1%	(Guizzardi 2011) ¹
Reintervention rate (all causes)	0-14%	(Guizzardi 2011) ¹ (Corriero 2013) ⁶ (Galarza et al. 2014) ⁷ (Feng et al. 2020a) ⁸ (Co-Minh 2021) ⁴
Conversion to fusion	0-8,3%	(Guizzardi 2011) ¹ (Corriero 2013) ⁶ (Galarza et al. 2014) ⁷ (Feng et al. 2020a) ⁸ (Co-Minh 2021) ⁴
Recurrence of Symptoms	5-13%	(Guizzardi 2011) ¹ (Corriero 2013) ⁶ (Galarza et al. 2014) ⁷ (Feng et al. 2020a) ⁸ (Co-Minh 2021) ⁴
Pain/ discopathy/ surgery at other levels of the spine	Up to 20%	(Co-Minh 2021) ⁴
Post operative pain at the instrumented level	40% transient	(Co-Minh 2021) ⁴
Intraoperative complications	1%	(Co-Minh 2021) ⁴
Device breakage	Not reported in clinical studies	NA

Table 6 Literature data – incident rates of IntraSPINE®

PMCF DATA

[PMCF Study 1] – 2012-2014 report 2805_INTRA-OBS_02-11-2-A_CSR_230709_V3-0_SIGNE-QM & corresponding Publication (Co-Minh 2021)

Title	<p>PMCF Study report : 3 years prospective study of surgical outcomes fo the Intraspine device in patients treated for degenerative spinal pathologies with lumbar disc disease</p> <p>Publication: Co-Minh D. Surgical outcomes of patients with degenerative lumbar disc disease post-IntraSPINE® device fixation: Three-year prospective study. Auctores Publishing LLC, editor. IJCCR. 2021 Aug 11;8(1):01–10.</p>
Type of study	Prospective, longitudinal, observational, cohort study multicenter, non-comparative
Device	IntraSPINE® Sterile devices for interspinous space with laminar support Intended to be used as a dynamic interspinous lumbar spacer with interlaminar support
Study period	From 26-11-2012 to dec 2014 + 3 years follow up (total 5 years)
Aims	To evaluate the results of patients with symptomatic Lumbar degenerative disc disease treated with an IntraSPINE® device and followed up over a 3-year period.
Evaluation criteria	<p>Primary outcome Success at 3 years, composite of improved ODI >15%, VAS >20% and radiological suces (disc height)</p> <p>Secondary outcome To describe the population of patients who have had implant placemen ODI 6m, 1y, 2y; VAS, Disc Height, neurological state, mobility (finger floor), waking time, time back to work, operating time, patient satisfaction To assess complications rate in per and postoperative</p>
Inclusion criteria	Inclusion diagnoses were: lumbar disc herniation (LDH), foraminal stenosis and facet joint syndrome

Non-inclusion criteria	Non-inclusion criteria were: aged < 18 years, allergy to one of the components of the implant, pregnancy, infected site and patients with known isthmic spondylolisthesis or spinal dysraphism.
Number of subjects analyzed and repartition	<p>231 patients were recruited and 180 completed the study</p> <ul style="list-style-type: none"> • Median age was 49 years • Male (58.9%) and female (41,1%) • Employed (62 %) • 83% of the patients had degenerative disc disease at a single level, with L4/5 being most commonly affected (59%)
Methods	<p>A prospective study was performed throughout the period from November 21, 2012 to December 20, 2017. It was conducted at 4 French clinics specializing in spinal surgery and involved one surgeon at each clinic. Patients were recruited from November 21, 2012 to December 1, 2014 (involving patients with imaging-confirmed Lumbar degenerative disc disease in whom conservative treatment was unsuccessful).</p> <p>Outcome measures were changes over baseline score on the Oswestry Disability Index (ODI), and low back and radicular pain assessed at 6, 12, 24 and 36 months postoperatively. Overall success, a composite outcome that included key safety and clinical considerations, was assessed.</p> <p>Secondary outcomes included satisfaction with symptoms, employment status and post-surgery medical interventions. To compare differences in longitudinal clinical score patterns over 36 months, a mixed-effect model ANCOVA with repeated measurements was performed, with adjustment for low back and radicular pain score and ODI score at baseline.</p>
Results	<p>A significant improvement in ODI score ($p=0.0597$), as well as in VAS (Visual Analogue Scale) scores for back ($p= 0.0228$) and leg pain ($p<0.0001$) was observed during the follow-up. For ODI score, the mean percentage decrease from inclusion to month 36 was 64.5%. These scores were respectively 66.2% for radicular pain and 46.4% for low back pain. In 73% of cases, surgery was considered successful. 89% of working patients returned to work and 68% of patients were very satisfied at month 12. Only four patients presented intraoperative complications.</p> <p>Among all adverse event reported in this study, none of them was related to a device deficiency. Removal of the device was observed in 12 cases.</p>

CONCLUSION

In summary, the findings of this study are promising and demonstrate that successful patient outcomes can be achieved using the IntraSPINE® device for a range of degenerative diseases, with improvements in both back/leg pain and disability, sustained at 3 years. Implantation of the IntraSPINE® device was straightforward and there were no device-specific complications. Use of the IntraSPINE® in the treatment of degenerative disease of the lumbar spine has been evaluated and discussed in published papers 27,31,43,45. The findings of this study coupled with other published research results suggest that IntraSPINE® may confer additional benefits in terms of back pain reduction and disability outcomes when used in conjunction with decompression or microdiscectomy in comparison to stand-alone decompressive procedures.

PMCF study – Interim clinical study report - Efficacy and safety of the inter-laminar device IntraSPINE® associated with discectomy in the treatment of lumbar disc herniation”. A single-blind, randomized, comparative, multicentre clinical trial vs. discectomy alone.

Title	<p>“Efficacy and safety of the inter-laminar device IntraSPINE® associated with discectomy in the treatment of lumbar disc herniation”.</p> <p>A single-blind, randomized, comparative, multicentre clinical trial vs. discectomy alone</p> <p>Interim clinical study report</p>	
Identification of the study	<p>CIP identification: 2834_INTREUR</p> <p>CIP date and version: V5-0 of 10/03/2021</p> <p>ID-RCB Number: 2018-A02185-50</p>	
Type of study	<p>Sub study (interim report) part of a Prospective study, multicentric, comparative, single-blind, randomized Study, Sponsored by Cousin Biotech</p> <p>Note : Only data of patients included in IntraSPINE® arm are analyzed, no success or failure criteria was defined.</p>	
Device	<p>IntraSPINE® Sterile devices for interspinous space with laminar support</p> <p>Intended to be used as a dynamic interspinous lumbar spacer with interlaminar support</p>	
Study period	<p>T1 2019 – T2 2023</p> <ul style="list-style-type: none"> • First inclusion: January 2019 • Last inclusion: June 2021 • Patient follow-up: 24 months • Follow-up of last patient: June 2023 	<p>Phase III</p>
Aims	<p>The primary objective is to assess at month 12 post-operatively the effect of IntraSPINE® device associated with discectomy on the degree of functional disability associated with lumbar pain following surgery for the treatment of lumbar disc herniation.</p> <p>Secondary objectives:</p> <ul style="list-style-type: none"> • To assess the effect of the IntraSPINE® device associated with discectomy in the treatment of lumbar disc herniation with regard to: <ul style="list-style-type: none"> • Recurrence of disc herniation during the study; • Evolution between inclusion, 1 month, 6 months, 12 months and 24 months post-surgery with regard to: <ul style="list-style-type: none"> • degree of functional disability associated with lumbar pain; • lumbar and radicular pain; • neurological and motor status. • To assess time to resumption of work post-surgery; • To assess rates of further surgery at the level of the implant during the study; • To assess adverse effects reported 	
Evaluation criteria	<p>Primary evaluation criterion:</p> <p>The degree of functional disability associated with lumbar pain will be measured using the Oswestry Disability Index (ODI) at 12 months and compared against the baseline data at inclusion.</p> <p>Secondary study evaluation criteria:</p> <ul style="list-style-type: none"> • Rate of re herniation (recurrence) • Oswestry Disability Index (ODI) at inclusion, 1 month, 6 months, 12 months and 24 months post –surgery; • Visual Analogic Scale (VAS) at inclusion, 1 month, 6 months, 12 months and 24 months post-surgery; 	

	<ul style="list-style-type: none"> Neurological and motor status will be assessed at L4, L5 and S1 levels (right and left sides) by measuring motor function, touch and prickling sensation at inclusion, 1 month, 6 months, 12 months and 24 months post-surgery. The time to resumption of work following surgery The revision surgery rate (Re intervention or further surgery) The number of adverse events (AE) and serious adverse events (SAE) The peroperative and short and long term postoperative complication rate Integrity of the implant: migration or wear
Inclusion criteria	<p>Disease-related criteria:</p> <ul style="list-style-type: none"> Single-segment lumbar disc herniation in L4-L5 or L5-S1, identified on imaging (MRI of lumbar spine) Median or posterolateral disc herniation Failure of properly conducted medical treatment which has not provided lasting symptom relief Absence of contraindications to surgical discectomy and implantation of the IntraSPINE®. <p>Population related criteria:</p> <ul style="list-style-type: none"> Subjects of both sexes aged over 18 years Who have given their free informed signed consent to participate in the study Patients who are able to respond to the questionnaires and who can communicate in the language of the study country Affiliated to a social security system or have rights from a social security system
Non-inclusion criteria	<p>Disease-related criteria:</p> <ul style="list-style-type: none"> Post-discectomy recurrence of hernia, regardless of level; Foraminal and extra-foraminal disc herniation; Cauda equina syndrome; Spondylolisthesis of any grade; Degenerative scoliosis $\geq 15^\circ$; Osteoporosis, Paget's disease; BMI > 30; Surgery required on more than one vertebral level; Endoscopic micro-discectomy prohibiting eventing insertion of IntraSPINE®; Chronic infection; Systemic or metabolic disorders; Active ongoing malignant disease (likely to interfere with the conduct of the study or induce vertebral lysis); Disease due to work accident. <p>Population-related criteria:</p> <ul style="list-style-type: none"> History of spinal surgery; Withdrawal of consent; Pregnancy; Breastfeeding; Participation in another clinical trial in the 3 months prior to the initial visit; Drug addiction; Predictable unavailability during the study. Patients deprived of their liberty or under legal guardianship.

	Treatment or device-related criteria: <ul style="list-style-type: none"> Allergy to any of the components of the medical device.
Number of subjects analyzed and repartition	<p>198 patients will be included over 24 months in both arm. For the moment, 92 patients have been included in IntraSPINE® arm.</p> <ul style="list-style-type: none"> Median age of 40.6 years Male (58.7%) and female (41.3%) Median duration of symptoms : 0.2 years 40.2% with a working activity (67.6 % with a sedentary work) Previous treatment: 90.1% drug therapy, 14.6% lumbar belt , 17.4% epidural infiltration, and 15.4% Physical therapy. 47 patients with L4-L5 herniations and 45 with L5-S1 herniations
Methods	<p>This is a sub study, part of a prospective, international, randomised, comparative, multicentre, superiority study in parallel groups with an assignment ratio of 1:1 and single-blind (patients are unaware of the group to which they have been randomised). Only data of patients included in IntraSPINE® arm are analysed. Each patient was followed for a period of 24 months. For the moment, the inclusions are not yet completed, and all the patients have not reached their full follow-up. 6 follow up visits were scheduled (expected surgery visit):</p> <ul style="list-style-type: none"> Visit 1: Inclusion (preoperative) Visit 2: Surgery. Visit 3: 1 month postoperatively Visit 4: 6 months postoperatively Visit 5 : 12 months postoperatively Visit 6 : 24 months postoperatively
Results	<p>→ ODI The mean 12 months follow-up score was 7.1 %, which is statistically better than the preoperative value of 55.9% (P < 0.0001). A trend of an overall and continuous decrease in the Oswestry score from inclusion up to 24 months of follow-up.</p> <p>→ LUMBAR – RADICULAR PAIN ANALYSIS A significant decrease in lumbar and radicular pain score was observed between inclusion and 24 months postoperatively. Radicular pain showed a greater improvement over time compared to low back pain (mean percentage decrease from inclusion to month 24 was respectively 98 % and 81%).</p> <p>→ NEUROLOGICAL AND MOTOR STATUS The MRC grading system provides the following grades: 0, paralysis; 1, only a trace or flicker of muscle contraction is seen or felt; 2, muscle movement is possible with gravity eliminated; 3, muscle movement is possible against gravity; 4, muscle strength is reduced, but movement against resistance is possible and 5, normal strength. Observation that most of patients have a normal strength whatever the follow up visit. Only a few patients presented a reduction in muscle strength at L5 before surgery, which returns to normal from V3.</p> <p>→ SENSORY ASSESSMENT: LIGHT TOUCH AND PIN PRICK Observation: whatever the level improvement in Light touch and pin prick test during follow-up.</p> <p>→ STRAIGHT LEG RAISING TEST Before surgery, 97.8% of patients had a positive straight leg-raising test while this percentage drops to 6.8% after 12 months follow-up and 0% after 24 months follow up.</p>

	<p>Regarding crossed straight leg raising test, positive test is observed in 49 patients (53.3%) before surgery, while 1.4% of patients present positive test 12 months after surgery and 0% after 24 months follow up.</p> <p>→ FINGER FLOOR DISTANCE Finger floor distance decreased from inclusion to 12 months follow up (from 40 cm to 10 cm), with a little increase from Month 12 to Month 24 .</p> <p>→ RETURN TO WORK Of the 15 working patients with a prescribed work stoppage, 3 (20%) returned to work after surgery. Median time to return to work was 86 days.</p> <p>→ ANALGESICS INTAKE The consumption of analgesics was reduced in all patients. With one patient taking analgesics 24 months postoperatively.</p> <p>→ SAFETY ANALYSIS 51 (55.4%) patients presented at least one AE, for a total of 121 adverse events reported. Among the AE, 9 were noted to had possible, probable or certain causal relationship with the IntraSPINE® device In 17 patients, at least one SAE was reported. Of a total of 20 reported SAEs, causal relationship with the IntraSPINE® device was reported for only 3 SAEs Five reinterventions were observed during the patients follow up, with 3 removal of the device and two replacements. For patient 016-03, the investigator declared a removal of the IntraSPINE®, which took place at the same time as the installation of the IntraSPINE® Suspicion of disc herniation recurrence was observed in one patient during patients follow up and one patient with disc herniation recurrence was observed.</p>
Limitations	<p>None of them was related to a device deficiency. 3 patients lost to follow-up among the 92 (90 who received IntraSPINE® device) patients included in the IntraSPINE® arm</p>

CONCLUSION

This analysis showed persistent benefit from surgery using IntraSPINE® between inclusion visit and 24 months follow up. The follow-up results are statistically significantly better than the preoperative status in every measurement.

The patients' pain was improved, with greater gains demonstrated for relieving leg pain. Oswestry scores dropped from a preoperative mean of 55.9 % to a mean of 2.4 % after 24 months follow up.

Regarding complication, one patient experienced device migration, one patient had a removal of IntraSPINE® for suspicion of re herniation and one patient had a removal of IntraSPINE® for re her-niation. Two patients had a replacement of IntraSPINE® device, one of them was because of right lombosciatic relapse and walking disorders. The 24-months follow-up of patients with lumbar disc herniation after implantation of IntraSPINE® showed very good clinical and functional results. Like-wise, the consumption of analgesics by patients has sharply decreased.

These preliminary results show good clinical performance, and the benefit-risk ratio seems favourable. However, it is necessary to wait until the end of the clinical trial to conclude on the main objective (Oswestry Disability Index).

[PMCF Survey, 2019] : Interim report Appendix B_IntraSPINE Survey Safety data and topping off_signed 20200103

Aim of the survey	On 23 November 2021, we sent this survey to surgeons who are users of IntraSPINE implant for many years (France and worldwide), to know if they have already implanted the device in L1/L2 and/or L2/L3 levels, and to collect clinical data for these cases.
Methods	The survey was sent on 23 November 2021 to surgeons. The selection of surgeons was made on surgeons with a high volume of IntraSPINE use to obtain statistically representative data with usage on all possible implant levels.
Results	On the date of 25.11.2021, only one (1) surgeon answered to this survey, in Brazil. The Brazilian surgeon who answered to this survey is a user of IntraSPINE since five (5) years and has implanted 120 IntraSPINE implants. The Brazilian surgeon who answered to this survey has implanted IntraSPINE in L1/L2 level four (4) times and did not experience any post-operative complications. The Brazilian surgeon who answered to this survey has implanted IntraSPINE in L2/L3 level twenty (20) times and did not experience any post-operative complications.
Other data	<p>Surgeon Profile The Brazilian surgeon who answered to this survey is a user of IntraSPINE since five (5) years and has implanted 120 IntraSPINE implants.</p> <p>Implantation in L1/L2 The Brazilian surgeon who answered to this survey has implanted IntraSPINE in L1/L2 level four (4) times and did not experience any post-operative complications.</p> <p>Implantation in L2/L3 The Brazilian surgeon who answered to this survey has implanted IntraSPINE in L2/L3 level twenty (20) times and did not experience any post-operative complications.</p>
CONCLUSION	
The IntraSPINE device has already been implanted in four (4) cases in L1/L2 level, and in twenty (20) cases in L2/L3 level. No complication has been reported.	

5.3.1. PMS DATA

Since launch in 2007, 92 customer complaints have been registered. The total number of customers complaints is very low compared to number of products sold: 0,089%.

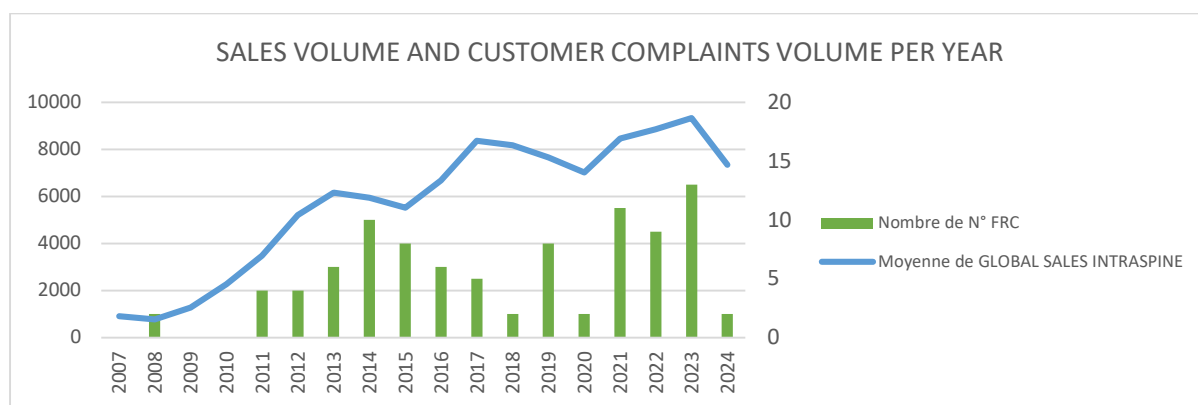
Figure 4 Sales volume and customer complaints volume per year

By considering “Device breakage / ligament” reported in 19 complaints since 2007, the repartition of this event has been evaluated.

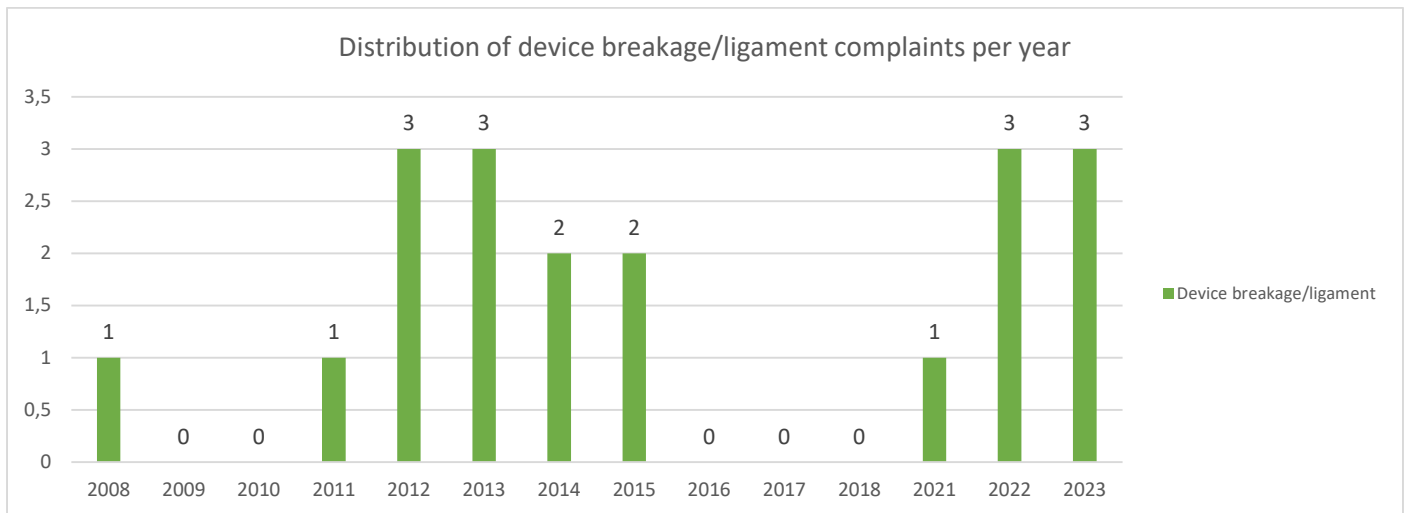
Figure 5 Yearly repartition of the device breakage/ligament event

No trend can be drawn regarding this event which are common and well-known complication of IntraSPINE®. Literature reports a rate inferior at 1,4% regarding the clinical data of IntraSPINE®

Since launch in 2007, 8 isolated incidents cases have been reported to competent authorities:

- 5 related to device breakage / wedge
- 1 related to allergy
- 1 related to discomfort/ pain
- 1 related to migration

The number of materiovigilance events is very low compared to the number of devices sold since 2007.

- ➔ No new type of incident has been reported since the last evaluation and all incidents correspond to known complications of this type of devices.
- ➔ No significant increase in the frequency and severity of incidents
- ➔ No trend can be observed.

5.4. AN OVERALL SUMMARY OF CLINICAL PERFORMANCE AND SAFETY

This part provides an overall summary of the clinical performance and safety supported by clinical data, post-market clinical follow-up data and post-market surveillance regarding IntraSPINE®.

The total experience with the device is:

- On the market : 102 855 devices sold since 2007
- In clinical Studies : 662 patients (840 devices)
- In users Surveys : > 2812 cases (incl 24 on L1-L3 levels, and 115 with Topping off technique)

Evaluation of the performances :

Performance outcome		Criteria / Objective (Threshold)	Demonstrated / How *
General performance outcome	Treat chronic low back pain	Pain scores: Improved score (min 1.5) ODI: Improved at least 15 points Success rate: Success or MACnab improved > 70% Quality of life (QoL) : Improved	Yes demonstrated by: <ul style="list-style-type: none"> - Clinical articles - PMCF studies - PMCF general methods - PMS data
	Full Motion preservation	Clinical Motion score : Motion scores as good as preoperatively	Yes demonstrated by: <ul style="list-style-type: none"> - Clinical articles - PMCF studies - Preclinical data
Specific performance outcome	Elastic device with cushioning effect	Preservation of function (ODI scores): Improvement of ODI	Yes demonstrated by: <ul style="list-style-type: none"> - Clinical articles - PMCF studies - Preclinical data
	Ligaments assist the posterior ligament during flexion	Clinical Motion score : Motion scores as good as preoperatively	Yes demonstrated by: <ul style="list-style-type: none"> - Clinical articles - PMCF studies - Preclinical data
	The thin ligament stabilizes the interlaminar device on its site	Radiological evaluation : Absence of device migration	Yes demonstrated by: <ul style="list-style-type: none"> - Clinical articles - PMCF studies - PMS data
	Respect of IAR of the segment	Radiological evaluation Clinical motion scores : Motion scores as good as preoperatively	Yes demonstrated by: <ul style="list-style-type: none"> - Clinical articles - PMCF studies - Preclinical data
	Wings: anti migration system	Radiological evaluation : Absence or rarity of device migration	Yes demonstrated by: <ul style="list-style-type: none"> - Clinical articles - PMCF studies - PMCF survey - PMS data
	Allows Minimal invasive surgery	Success of minimally invasive access / surgery duration : Usual surgery duration	Yes demonstrated by: <ul style="list-style-type: none"> - Clinical articles - PMCF studies - PMCF general methods - PMS data
	Fast and simple procedure (distraction and optimal positioning of the device)	Easiness of use Duration of surgery : No threshold in SOTA Comparison with expected surgery duration	Yes demonstrated by: <ul style="list-style-type: none"> - Clinical articles - PMCF studies - PMCF general methods - PMS data

Table 7 Synthetic table of performances

* See the table 5 Summary of performances (based on clinical literature), PMCF Studies, PMCF survey and PMS data detailing the demonstration of the clinical performance and benefits related to Arthropathic (DDD) facet syndrome, Disk Hernia / degenerative discopathy and Soft stenosis.

Evaluation of the benefits

The benefits for the patient are the following:

- Positive impact on clinical outcome: Less pain and less disability
- Patient's quality of life: Improvement of the quality of life

According to the clinical data, there is a clear impact on the quality of life for the patients as illustrated by a decrease of pain and disability (Improvement of mean scores in all studies 73-93% patients). Consequently, there is another benefit with public health impact as the percentage return to work is improved (Variable 20-60%).

Evaluation of the clinical risk of devices

The risks assessment for IntraSPINE® is based on:

- Device related incidents observed in post-market surveillance: See the section 5.3.3 PMS data
- Device / procedure incident identified in the clinical studies: See the table 4 Assessment of the risk associated with the use of IntraSPINE®

It is of note that no difference in the risk rates has been identified depending on the clinical situation.

Evaluation of the acceptability of the benefits/risk profile

Once the following have been examined:

- the benefits of the product when it is used following the recommendations of the manufacturer, demonstrated through clinical and performances for the following situation as described in Section 5.3 Summary of clinical data, Table 5 Summary of performances and Table 7 Synthetic tables of performances).
- the current state of the art, including available alternatives, all surgical with comparable B/R profiles taking in account that surgery is indicated only when conservative measures have failed with :
 - comparable surgical risks (invasive or mini-invasive spine surgeries) and complications rates
 - comparable outcomes and success rates, none of them having demonstrated any superiority
 - comparable recurrence and reintervention rates, none of them having demonstrated any superiority
- the acceptability of residual risks and undesirable side effects identified in view of the above benefits and the state of the art

It can be concluded that the benefit risk of the device is acceptable and unchanged from previous evaluations.

5.5. ONGOING OR PLANNED POST MARKET CLINICAL FOLLOW-UP

To ensure a post-market clinical follow-up and updating the clinical data, some activities are planned annually with general and specific methods:

- **General methods:**
 - Screening of scientific literature and other sources
 - Collect and analysis of data acquired clinical experience

- Collect and analysis of surveys from health care professional
- Analysis of post-market surveillance reports
- Review of case which may reveal misuse or off-label use
- **Specific methods**
 - Interventional PMCF clinical investigation: INTEREUR STUDY "Efficacy and safety of the inter-laminar device IntraSPINE® associated with discectomy in the treatment of lumbar disc herniation ».
Study concerning 8 centers with France (2 sites), Spain (3 sites), Georgia (3 sites)
The final report is planned for Q1 2024. See section 5.3.2 PMCF DATA for details

6. POSSIBLE DIAGNOSTIC OR THERAPEUTIC ALTERNATIVES

Regarding the clinical condition defined for IntraSPINE®, the possible diagnostic or therapeutical alternatives are the following:

Non-surgical approaches

Non-surgical options include drugs, physiotherapy, spinal injections, lifestyle modification, and multidisciplinary rehabilitation. It is generally admitted in the guidelines that these approaches should always be considered as first choice given the limited level of demonstration of the benefit of surgical technique on the long term.

Nevertheless, when all applicable non-surgical approaches have been tried and failed for at least several months, invasive approaches can be proposed, with a careful weighing of the benefit risk of the intervention for the patient.

Invasive / Surgical approaches

Degenerative intervertebral disc disease and/or articular facets confirmed by history and radiographic studies

In the majority of cases, spinal disc herniation articular facets syndrome doesn't require surgery. Surgery may be useful in those with a herniated disc that is causing pain and neurological symptoms in the lower limbs. Several techniques are described in the literature:

- Fusion surgery
- Total disc arthroplasty
- Other Biological approaches to prevent degeneration of discs (Stem cell therapy)

Spinal stenosis

Almost all patients who seek treatment for their claudication symptoms are started off with a conservative regimen. Non-surgical treatment options include rest, analgesics and anti-inflammatory, physical therapy, steroid injections and alternative approaches such as acupuncture.

Conservative method of treatment is generally followed for a period of 6 months. Upon the failure of conservative measures, surgical treatment is recommended.

Surgical options have the primary goal of decompressing the neural components that are being aggravated, which should relieve the symptoms. Several techniques are described in the literature:

- Open laminectomy
- Lumbar spinal fusion
- Biportal endoscopic spinal surgery
- Interspinous spacers

Dynamic stabilization spinal surgery using interspinous devices (ISD) (Technique of IntraSPINE®)

These techniques are less invasive and have been developed to preserve spinal integrity and to minimize tissue damage by limiting bony decompression and avoiding removal of the midline structures (i.e. spinous process, vertebral arch and interspinous and supraspinatus ligaments).

The hypothesis behind dynamic stabilizations is to control abnormal motions, so that greater physiological load transmissions can relieve pain and prevent adjacent segment degenerations. It is expected that once normal motion and load transmission is achieved, the disc may repair itself, unless the degeneration has progressed beyond repair. By stabilizing vertebral motion, pain may be minimized compared with conservative treatment.

- *Dynamic stabilization with ISD versus Arthrodesis (Spinal Fusion)*

The complete fusion has the significant potential to accelerate adjacent level degeneration which is minimized by the dynamic stabilization (or total disk replacement). Clinical data support the pathophysiological assumption of a decrease of the adjacent segment degeneration in the presence of ISD.

Dynamic stabilization with ISD	Arthrodesis (Spinal Fusion)
Mini invasive surgery: Surrounding muscles and ligaments preservation	Open surgery No preservation of surrounding structures
Better outcomes: disability, operation time, blood loss and hospitalization	Current one of the gold standard surgical options
Prevent adjacent segment degeneration (ASD)	Fusion of intervertebral levels = loss of segmental mobility leading to ASD phenomenon
Used in topping off technique to prevent ASD phenomenon	
Adapted for elderly patients	
ISD can be easily removed and converted to laminectomy if symptoms reemerge	
Similar scoring for VAS (pain)	
Similar rate of adverse events and revision surgery	

- *Decompression act with ISD versus Decompression act standalone*

The perceived advantages of the ISD compared to the current gold standard treatment seem to be a minor invasiveness, a reversibility, the not requirement of removal of posterior elements and better outcomes in term of disability.

Decompression act with ISD	Decompression act standalone
Mini invasive surgery : Surrounding muscles and ligaments preservation	Open surgery No preservation of surrounding structures
No requiring concomitant surgical decompression	Current one of the gold standard surgical options
Less perioperative complication than decompression act alone	Epidural space opening is known to routinely produce epidural adhesions around dural sac which can cause symptomatic problems
Operation time shorter, Perioperative blood loss lower, Shorter hospitalization time	/
Better effect on clinical outcomes (ODI, VAS, QoL)	Less reoperation rate
ISD can be easily removed and converted to laminectomy if symptoms	/

Emerge	
Higher cost	/

7. SUGGESTED PROFILE AND TRAINING FOR USERS

The device should be implanted only by a qualified surgeon (orthopaedic surgeons and neurosurgeons), with appropriate knowledge of anatomy and spinal surgery, who must be extremely familiar with the device, its intended use, and the surgical technique to be performed.

Before using the IntraSPINE® device, the surgeon must be trained to the surgical technique. For this purpose, Cousin Biotech has implemented several resources with the cooperation of surgeons who are experts in the use of IntraSPINE®:

- Theoretical training through the surgical technique available for download on the Cousin Biotech website or sent to surgeons on request. This document describes step by step how to use IntraSPINE®. It has been written and validated by experts. Moreover, virtual supports with 3D video on the use of the device are available on request.
- Peers to peers training or cadaver courses with expert surgeons (Annual sessions organized by Cousin Biotech).

In addition, dedicated instruments must be used for its implantation as described in the surgical technique for each step (preparation, decontamination and cleaning, sterilization).

The devices are only to be used in the operating room of a hospital setting.

8. REFERENCE TO ANY HARMONISED STANDARDS AND COMMON SPECIFICATIONS APPLIED

Hereafter, the list of the standards claimed for the Sterile devices for interspinous space with laminar support, showing the standards harmonised according to Regulation 2017/745 and the state of the art of standards.

<u>Standards title</u>	<u>Date</u>
EN ISO 13485:2016 /AC 2018 -- Medical devices - Quality management systems - Requirements for regulatory purposes	April 2016
EN ISO 13485:2016 /A11 -- Medical devices - Quality management systems - Requirements for regulatory purposes	September 2021
NF EN ISO 14630: Non-active surgical implants - General Requirements	February 2013
NF EN ISO 9001 Quality Management systems - Requirements	October 2015
NF EN ISO 7153-1: Surgical Instruments – Metallic Materials – Part 1: Stainless steel	December 2016
ISO 19227: Implants for surgery – Cleaning of orthopaedic implants – General requirements	March 2018
NF EN 556-1 -- Sterilization of medical devices - Requirements for medical devices to be designated "STERILE" - Part 1: Requirements for terminally sterilized medical devices	July 2024

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NF EN ISO 11137-1:2015/A2 Sterilization of health care products -- Radiation -- Part 1: Requirements for development, validation and routine control of a sterilization process for medical devices	November 2019
NF EN ISO 11137-2: Sterilization of health care products — Radiation — Part 2: Establishing the sterilization dose	May 2020
ISO 11607-1 -- Packaging for terminally sterilized medical devices - Part 1: requirements for materials, sterile barrier systems and packaging systems	January 2020
ISO 11607-2 -- Packaging for terminally sterilized medical devices - Part 2: Validation requirements for forming, sealing and assembly processes	January 2020
NF EN ISO 11737-1: Sterilization of health care products — Microbiological methods —Part 1: Determination of a population of microorganisms on products	January 2018
EN ISO 11737-2 : Sterilization of health care products — Microbiological methods — Part 2: Tests of sterility performed in the definition, validation and maintenance of a sterilization process	May 2020
NF EN ISO 14937 Sterilization of health care products — General requirements for characterization of a sterilizing agent and the development, validation and routine control of a sterilization process for medical devices	December 2009
NF EN ISO 11140-1 Sterilization of health care products - Chemical indicators - Part 1: general requirements.	February 2015
ISO 14155 -- Clinical investigation of medical devices for human subjects - Good clinical practice	July 2020
NF EN ISO 10993 – 1: -- Biological evaluation of medical devices. Part 1: Evaluation and testing within a risk management process	July 2010
NF EN ISO 10993-1: Biological evaluation of medical devices. Part 1: Evaluation and testing within a risk management process	December 2020
NF EN ISO 10993- 3 -- Biological evaluation of medical devices. Part 3: Tests for genotoxicity, carcinogenicity and reproductive toxicity	December 2014
NF EN ISO 10993-5 -- Biological evaluation of medical devices. Part 5: Tests for in vitro cytotoxicity	July 2010
NF EN ISO 10993-6 -- Biological evaluation of medical devices. Part 6: Tests for local effects after implantation	April 2017
NF EN ISO 10993-10 -- Biological evaluation of medical devices. Part 10: Tests for irritation and skin sensitization	February 2023
NF EN ISO 10993 -11 :2018 -- Biological evaluation of medical devices. Part 11: Tests for systemic toxicity	May 2018
NF EN ISO 10993-12 -- Biological evaluation of medical devices - Part 12: Sample preparation and reference materials	June 2021
NF EN ISO 10993-17 -- Biological evaluation of medical devices - Part 17: Establishment of allowable limits for leachable substances	November 2023
NF EN ISO 10993-18 -- Biological evaluation of medical devices. Part 18 : Chemical characterization of materials	May 2020
ISO/TR 15499 Biological evaluation of medical devices—Guidance on the conduct of biological evaluation within a risk management process	January 2017
ISO/TS 10993-19 - Biological evaluation of medical devices	March 2020
ISO 15223-1: Medical Devices -- Symbols to be used with medical device labels, labelling and information to be supplied -- Part 1: General Requirements	September 2021

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ISO 20417 : Dispositifs médicaux -- Informations à fournir par le fabricant	May 2021
NF EN ISO 14971: Medical devices -- Application of risk management to medical devices	January 2013
ISO 14971 : Medical devices - Application of risk management to medical devices	December 2019
NF EN ISO 14971:2019/A11 Medical devices - Application of risk management to medical devices	December 2021
IEC 62366-1: Medical devices -- Part 1: Application of usability engineering to medical devices	December 2015
NF EN IEC 62366-1/A1:2020 Medical devices - Part 1: Application of usability engineering to medical devices - Amendement 1	August 2020
NF EN ISO/IEC 17025: 2017 General requirements for the competence to carry out tests and/or calibrations, including sampling. It covers testing and calibration performed using standard methods, non-standard methods, and laboratory-developed methods	December 2017
NF EN 868-5 -- Packaging for terminally sterilized medical devices - Part 5:Sealable pouches and reels of porous materials and plastic film construction - Requirements and test methods	December 2018
NF EN ISO 14644-1: Cleanrooms and associated controlled environments. Part 1: classification of air cleanliness	February 2016
NF EN ISO 14644-2: Cleanrooms and associated controlled environments. Part 2: monitoring to provide evidence of cleanroom performance related to air cleanliness by particle concentration	February 2016
NF EN ISO 14644-3: Cleanrooms and associated controlled environments. Part 2: test methods	October 2019
NF ISO 2859-1/A1: Sampling procedures for inspection by attributes — Part 1:Sampling schemes indexed by acceptance quality limit (AQL) for lot-by-lot inspection	April 2000 / A1 : September 2011
ISO 12891-1: Retrieval and analysis of surgical implants - Part 1: Retrieval and handling	July 2015
NF EN ISO 19011 - Guidelines for auditing management systems	January 2012
ISO 19011 - Guidelines for auditing management systems	July 2018
NF EN ISO 5832-2 :2018 - Implants for surgery -- Metallic materials -- Part 2: Unalloyed titanium	May 2018
NF EN ISO 5832-3 :2016 - Implants for surgery. Metallic materials Part 3: Wrought titanium 6 aluminium 4 vanadium alloy	December 2021
NF S99-401 - Medical devices – Medical grade silicone elastomers	March 1994
ISO 14949 - Implants for surgery – Two part addition-cure silicone elastomers	October 2001
ISO 7153-1 Surgical Instrument – Materials – Part 1: metals	December 2016
EN ISO 2060 Textiles - Yarn from packages - Determination of linear density (mass per unit length) by the skein method	June 1995
EN ISO 2061 Textiles - Determination of twist in yarns — Direct counting method	2015
NF EN ISO 2062 Textiles- Varns from packages - Determination of single-end breaking force and elongation at break	January 2010
NF EN 14621 Textiles - Multifilament yarns - Methods of test for textured or non-textured filament yarns	2006
DIN 54278-1 Testing of textiles - Coatings and attendant materials - Part 1: Determination of materials soluble in organic solvents	1995

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NF EN ISO 13934-1 Textiles - Tensile properties of fabrics - Part 1: determination of maximum force and elongation at maximum force using the strip method	July 2013
ISO 3801 - Textiles - Woven Fabrics - Determination of Mass per Unit Length and Mass per Unit Area	September 1977
NF EN 12127 Textiles - Fabrics - Determination of mass per unit area using small samples	March 1998
ASTM F136: Standard Specification for Wrought Titanium-6Aluminum-4Vanadium ELI (Extra Low Interstitial) Alloy for Surgical Implant Applications	January 2013
ASTM D4169 -- Standard Practice for Performance Testing of Shipping Containers and Systems	2016
ASTM D4332 -- Standard Practice for Conditioning Containers, Packages, or Packaging Components for Testing	2014
ASTM D642 -- Standard Test Method for Determining Compressive Resistance of Shipping Containers, Components, and Unit Loads	2015
ASTM D5276 -- Standard Test Method for Drop Test of Loaded Containers by Free Fall	2015
F1929 -- Standard test Method for Detecting Seal Leaks in Porous Medical Packaging by Dye Penetration	2015
F1980 -- Standard Guide for accelerated Ageing of Sterile Barrier Systems for Medical Devices	2016
ASTM F1886 -- Standard Test Method for Determining Integrity of Seals for Medical Packaging by Visual Inspection	2016
ASTM D2240 -- Standard Test Method for Rubber Property—Durometer Hardness	August 2015
MEDDEV 2.7.1 Rev.4 Clinical Evaluation : A Guide For Manufacturers and Notified Bodies	June 2016
MEDDEV 2.7.2 Rev.2 Guidelines for Competent Authorities for making a validation/assessment of a clinical investigation application under directives 90/385/EEC and 93/42/EC	September 2015
MEDDEV 2.7.3 Rev.3 Clinical investigations : serious adverse event reporting – SAE reporting form	May 2015
MEDDEV 2.7.4 Rev.3 Guidelines on Clinical investigations: a guide for manufacturers and notified bodies	December 2010
MEDDEV 2.12.1 Rev.8 Guidelines on a Medical Devices Vigilance System	January 2013
NBOG's Best Practice Guide 2014-3 Guidance for manufacturers and Notified Bodies on reporting of Design Changes and Changes of the Quality System	2014
MHRA Guidance - Guidance on the regulations for electronic instructions for use of medical devices	February 2015
ISO TR 20416 2020 Post Market Surveillance for Manufacturers	July 2020
MDCG 2020-10/1 Safety reporting in clinical investigations of medical devices under the Regulation (EU) 2017/745	October 2022
MDCG 2020-10/2 Clinical Investigation Summary Safety Report Form v1.0	October 2022
MDCG 2020-13 Clinical evaluation assessment report template	July 2020

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MDCG 2020-8 Guidance on PMCF evaluation report template	April 2020
MDCG 2020-7 Guidance on PMCF plan template	April 2020
MDCG 2020-6 Guidance on sufficient clinical evidence for legacy devices	April 2020
MDCG Clinical Evidence needed for medical devices previously CE marked under Directives 93/42/EEC or 90/385/EEC. A guide for manufacturers and notified bodies	April 2020
MDCG 2019-9 Summary of safety and clinical performance	March 2022
MDCG Registration of legacy devices in EUDAMED	April 2019
Timeline for registration of device data elements in EUDAMED	April 2019
MDCG Guidance Document implant card on the application of Article 18 Regulation (EU) 2017/745 on medical devices	March 2020
MDCG The EMDN - The Nomenclature of use in EUDAMED	January 2020
MDCG The CND Nomenclature - Background and general principles	January 2020
Guidance on Basic UDI-DI and changes to UDI-DI	June 2020
MDCG guiding principles for issuing entities rules on basic UDI-DI	June 2020
Guidance on harmonised administrative practices and alternative technical solutions until EUDAMED is fully functional	May 2021

BIBLIOGRAPHY

- ¹ Guizzardi G. IntraSPINE, an interlaminar, not interspinous, posterior motion preservation device in Lumbar DDD: indications and clinical results (over 2 year follow-up). *Revista Latino-Americana de Neurocirugia*. 2011 Nov;18:13–20
- ² Guizzardi G, Morichi R. Clinical Results with IntraSPINE®. *EC Orthopaedics*. 2015;2:101–6.
- ³ Gelder CL, Drozda M, Spink G. LUMBAR MICRODISCECTOMY WITH INTRASPINE® – A CASE SERIES. *Coluna/Columna*. 2020 Jul;19(3):201–4.
- ⁴ Co-Minh D. Surgical outcomes of patients with degenerative lumbar disc disease post-IntraSPINE® device fixation: Three-year prospective study. *Auctores Publishing LLC, editor. IJCCR*. 2021 Aug 11;8(1):01–10.
- ⁵ Bae J, Lee SM, Lee S-H, Shin S-H, Kim H-J, Kim KH. The Likelihood of Reaching Substantial Clinical Benefit After an Interlaminar Dynamic Spacer for Chronic Low Back Pain: A Clinical and Radiologic Analysis of a Prospective Cohort. *World Neurosurgery*. 2017 May;101:589–98.
- ⁶ Corriero OV. Lumbar Herniated Disc Treated by Microdiscectomy Alone or Microdiscectomy Plus an Interlaminar Shock Absorbing Device: Retrospective Study with Minimum 3-Years Follow-Up. *J Spine [Internet]*. 2013 [cited 2019 Jul 24];03(02). Available from: <http://www.omicsgroup.org/journals/lumbar-herniated-disc-treated-by-microdiscectomy-alone-or-microdiscectomy-plus-an-interlaminar-shock-absorbing-device-2165-7939-3-159.php?aid=23954>

⁷ Galarza M, Gazzeri R, De la Rosa P, Martínez-Lage JF. Microdiscectomy with and without insertion of interspinous device for herniated disc at the L5–S1 level. *Journal of Clinical Neuroscience*. 2014 Nov;21(11):1934–9.

⁸ Feng S, Fan Z, Li X, Lin J, Yang Y, Fei Q. Combination of Intraspine Interlaminar Device With Lumbar Discectomy in Patients With L5/S1 Disc Herniation [Internet]. In Review; 2020a Sep. Available from: <https://www.researchsquare.com/article/rs-64740/v1>

⁹ Bistazzoni S, Angelis MD, D'ercole M, Chiaramonte C, Carotenuto A, Cardarelli G, et al. Evaluation of Effect of Posterior Dynamic Stabilization IntraSPINE System on Sagittal Spinal Balance using EOS® X-Ray Imaging System. *J Neurol Neurophysiol* [Internet]. 2017 [cited 2019 Jul 24];08(04). Available from: <https://www.omicsonline.org/open-access/evaluation-of-effect-of-posterior-dynamic-stabilization-intraspine-system-on-sagittal-spinal-balance-using-eos-xray-imaging-system-2155-9562-1000439.php?aid=92605>

SECTION 2: SSCP FOR PATIENTS

This Summary of Safety and Clinical Performance (SSCP) is intended to provide public access to an updated summary of the main aspects of the safety and clinical performance of the device. The information presented below is intended for patients or lay persons. A more extensive summary of its safety and clinical performance prepared for healthcare professionals is found in the first part of this document.

The SSCP is not intended to give general advice on the treatment of a medical condition. Please contact your healthcare professional in case you have questions about your medical condition or about the use of the device in your situation. This SSCP is not intended to replace an Implant card or the Instructions For Use to provide information on the safe use of the device.

1. DEVICE IDENTIFICATION AND GENERAL INFORMATION

1.1. DEVICE TRADE NAME(S)

The family identified and concerned by the SSCP is Sterile devices for interspinous space with laminar support, included: IntraSPINE®.

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1.3. BASIC UDI-DI

376018557INTRASPINEIMP47

1.4. YEAR WHEN THE FIRST CERTIFICATE WAS ISSUED COVERING THE DEVICE

The CE marking certificate number of IntraSPINE is FR19/81843462 under Directive 93/42/CEE. The device INTRASPINE® is CE marked since March 2007 (93/42/EEC).

The CE certificate was obtained for the RCBINTRAXX and BIOINTRAXX references, in accordance with European Regulation 2017/745, on 2025-05-09. The certificate numbers are as follows:

- EU Quality Management System Certificate MDR 00019-A
- EU Technical Documentation Assessment Certificate MDR 00019-B

Note: The reference variants RCBINTRAXX and INTRACALXX are identical (INTRACALXX is the name of the product under directive 93/42/EEC, RCBINTRAXX is the name under regulation 2017/745 UE). The creation of these new references (RCBINTRAXX) was made for commercial and logistic reasons in view of the certification under Regulation 2017/745. The decision was made to delete the RCBINTRAXX references and replace them with the INTRACALXX references, which were already used under Directive 93/42/EEC.

2. INTENDED USE OF THE DEVICE

2.1. INTENDED PURPOSE

IntraSPINE® is intended to be used as a dynamic interspinous lumbar spacer with interlaminar support. This device belongs to the non-fusion spinal products offering an alternative to arthrodesis surgery.

2.2. INDICATIONS AND INTENDED GROUPS

The IntraSPINE® is an interspinous with laminar support prosthesis for discal or facet assistance. This device is indicated for use from L1-S1 in skeletally mature patients with at least moderate impairment in function and who experience relief from symptoms of leg/buttocks/groin pain in flexion, this being either associated or not with back pain, and who have undergone at least 6 months of non-operative treatment. The indications of the IntraSPINE® prosthesis are as follows:

- Arthropathic facet-syndrome
- Foraminal stenosis
- Degenerative discopathy

2.3. CONTRAINDICATIONS

The IntraSPINE® prosthesis is contraindicated under the following circumstances:

- Allergy to silicone or one of its other components
- Growing child
- Infected site
- Pregnant woman

The following factors could compromise successful implantation:

- Severe osteoporosis
- Major deformations of the spinal column
- Local bone tumours
- Systemic or metabolic disorders
- Infectious diseases
- Obesity
- Drug addictions
- Intense physical activity, including competitive sports or hard labour
- Surgery at more than two levels with IntraSPINE® prosthesis
- Spondylolisthesis from Grade 2 to 4
- Intervertebral disc with Pfirrmann classification Grade 5
- S1 spinous process length inferior to 18mm when implanted in L5/S1

3. DEVICE DESCRIPTION

3.1. DESCRIPTION OF THE DEVICE AND MATERIALS / SUBSTANCES IN CONTACT WITH PATIENT TISSUES

3.1.1. GENERAL DESCRIPTION

The IntraSPINE® prosthesis is an interlaminar medical device that comprises a dimethyl siloxane (silicone) wedge (available in several sizes²) adapted to the intervertebral space and one or two ligaments in polyethylene terephthalate.

The wedge is covered with woven polyethylene terephthalate with an area covered by sheeting on the front face in the medullary zone.

Tension is applied to the ligament with a titanium loop (TA6V) or with a crimping system in titanium (T40) (depending on the choice of ligament used).

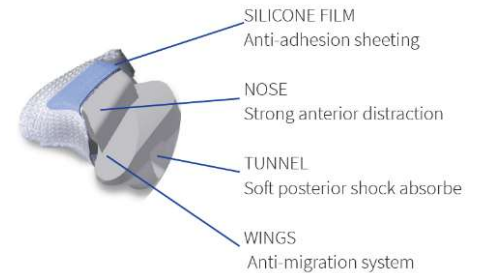
Figure 6 - IntraSPINE® wedge

Figure 7 - IntraSPINE® device (BIOINTRAXX variant)

The IntraSPINE® is an elastic support device with a cushioning effect. Its interlaminar implantation enables to be closed to the centre of rotation, thereby producing the cushioning effect and providing tension on the inter-spinous ligamentary structures due to the elasticity of the wedge.

The principal function of the ligament surrounding the upper and lower vertebrae in relation to the area receiving the prosthesis is to assist the posterior ligament during flexion.

The second ligament stabilizes the interlaminar device on its site.

The intended use of the medical device IntraSPINE® is to treat the Degenerative (intervertebral) disc disease (DDD) and its consequences.

IntraSPINE® is an invasive and implantable sterile device, sterilized by gamma radiation for single use.

² The wedge is available in several sizes (8mm, 10mm, 12 mm, 14 mm and 16mm) to fit to patient 's anatomy

3.1.2. LIST OF REFERENCES

Hereafter, the references, and their description:

PRODUCT CODE	PRODUCT SHAPE	DESCRIPTION
INTRACAL08		Wedge 8mm covered by a knitting +1 ligament with a titanium loop
INTRACAL10		Wedge 10mm covered by a knitting +1 ligament with a titanium loop
INTRACAL12		Wedge 12mm covered by a knitting +1 ligament with a titanium loop
INTRACAL14		Wedge 14mm covered by a knitting +1 ligament with a titanium loop
INTRACAL16		Wedge 16mm covered by a knitting +1 ligament with a titanium loop
BIOINTRA08		Wedge 8mm covered by a knitting +1 ligament with a titanium loop + 1 needled ligament with a hardened loop + crimp device on nylon wire
BIOINTRA10		Wedge 10mm covered by a knitting +1 ligament with a titanium loop + 1 needled ligament with a hardened loop + crimp device on nylon wire
BIOINTRA12		Wedge 12mm covered by a knitting +1 ligament with a titanium loop + 1 needled ligament with a hardened loop + crimp device on nylon wire
BIOINTRA14		Wedge 14mm covered by a knitting +1 ligament with a titanium loop + 1 needled ligament with a hardened loop + crimp device on nylon wire
BIOINTRA16		Wedge 16mm covered by a knitting +1 ligament with a titanium loop + 1 needled ligament with a hardened loop + crimp device on nylon wire

Table 8 References of Sterile Devices for Interspinous Space with Laminar Support

3.1.1. MATERIALS AND CONTAINED SUBSTANCES

Duration of use or contact with the body

Long term implantation - Permanent Implant > 30 days

Materials or substances in contact with the patient tissues

The body part on which the device acts or with which it interacts is lumbar vertebrae.

BRAND NAME	PART OF THE DEVICE	COMPONENTS	RAW MATERIALS (Commercial name)
IntraSPINE®	Wedge	Nude Wedge	Dimethyl siloxane MED4765
		Glu	Dimethyl siloxane MED20000
		Sheeting	Dimethyl siloxane MED4750
		Sewing thread	Polyethylene terephthalate

Ligaments	Knitting	Polyethylene terephthalate
	Braid	Polyethylene terephthalate
	Sewing thread	Polyethylene terephthalate
	Loop	Titanium TA6V

Table 9 Materials or substances in contact with the patient tissues for IntraSPINE®

Other substances

A medicinal substance (including a human blood or plasma derivative)	No
Tissue(s) or cells of human or animal origin, or their derivatives	No
Substances or combinations of substances that are absorbed by or locally dispersed in the human body	No
Materials incorporated into the device that contain or consist of CMR (carcinogenic, mutagenic or toxic to reproduction) substances or endocrine-disrupting substances	No
Materials that could result in sensitisation or an allergic reaction by the patient or user	No

3.2. INFORMATION ABOUT MEDICINAL SUBSTANCES IN THE DEVICE, IF ANY

Not applicable. The device does not contain any medicinal substances.

3.3. DESCRIPTION OF HOW THE DEVICE IS ACHIEVING ITS INTENDED MODE OF ACTION

IntraSPINE® has been designed to be implanted in the interlaminar area.

From its implantation in the interlaminar space, the device is closer of the AIR (Axis instantaneous of rotation), and this allows maintain a better range of motion.

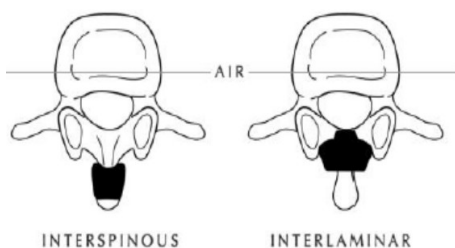
Figure 8 - Interspinous versus interlaminar implantation

This design allows three functionalities:

- The nose (anterior part) is rigid and positioned between laminae in order to recover a normal disc height.
- The tunnel in the posterior part allows the cushioning effect between the spinous processes.
- The wings (posterior part) represent the security of the device because they are developed to avoid any anterior migration.

Specific dimensions have been established during the design development of the prosthesis to perfectly match the anatomy of the lumbar segment and to ensure the claimed clinical performances.

During the surgery, a distraction is made by moving the 2 spinous processes with a specific ancillary. This helps the insertion of IntraSPINE[®] implant in the interlaminar area. The nose of the prosthesis is placed against two laminae and the body containing the tunnel is between upper and lower spinous process. Then, the ligament surrounding the upper and lower vertebrae maintains the spinous process in their natural position and consequently maintains the compression on the nose. The ligament brings additional safety against potential posterior migration.

3.4. DESCRIPTION OF ACCESSORIES, IF ANY

Special REUSEABLE ancillaries are used to implant this device: 1 large bone forceps, distraction forceps, 5 trial devices (5 sizes), 1 left and 1 right wire guides, 1 implant holder, 1 impactor, 1 ligament tightener and 1 crimp tool (only for BIOINTRAXX).

They are sold as a complete set: reference KITACINTRA for INTRACALXX references or KITBIOINTR for BIOINTRAXX references.

4. RISKS AND WARNINGS

Contact your healthcare professional if you believe that you are experiencing side-effects related to the device or its use or if you are concerned about risks. This document is not intended to replace a consultation with your healthcare professional if needed.

4.1. HOW POTENTIAL RISK HAVE BEEN CONTROLLED OR MANAGED

To control and manage potential risks, the manufacturer complies with applicable regulations, standards and directives. The manufacturer applies risk management, clinical evaluation, post-market surveillance and vigilance processes. This includes continuous review of clinical data and feedback on device use.

4.2. REMAINING RISKS AND UNDESIRABLE EFFECTS

Per instructions for use NOT293 (INTRACALXX) and NOT310 (BIOINTRAXX), complications associated with using the device under evaluation are:

Complications likely related to the device

- Infection of the prosthesis
- Device breakage (rupture of the ligament, fixed wing breakage, or knitting tearing)
- Uncrimping of the needle
- Device loosening permanent elongation of the ligament
- Device migration displacement of the wedge
- Spondylolisthesis to the dynamically stabilised segment
- Persistent or worsening pain attributed to the dynamically stabilised level, requiring revision surgery with interbody fusion at that level
- Spinous process fractures, either intraoperatively, post-operatively, or upon device removal (removal due to infection), which can be associated with implant dislocation
- Late reoperation due to recovery failure
- Disability
- Discomfort
- Wrong level implantation, with inappropriate or inaccurate positioning (posterior or wrong level)
- Seroma/swelling around the device

- Inflammatory reaction, allergy, or foreign body reaction
- In certain cases, the medical instrument had to be removed, with pedicle screw fusion performed.

Known complications of the procedure (if applicable)

- Pseudo-meningocele, fistula, dural tear, persistent CSF leakage, and meningitis
- Loss of neurological function
- Cauda equina syndrome, neuropathy, neurological deficits, arachnoiditis, muscle loss, compression around nerves, and pain
- Urinary retention, loss of bladder control, or other urological system compromises
- Fracture, bone loss, or decreased bone mineral density
- Herniated nucleus pulposus and disc disruption or degeneration either at, above, or below the surgery level, with adjacent segment degeneration
- Non-union or pseudo-arthritis, delayed union, or mal-union
- Cessation of any potential growth of the operated spine portion
- Either loss of or increase in spinal mobility or function
- Haemorrhage, haematoma, oedema, embolism, stroke, excessive bleeding, phlebitis, wound necrosis, wound dehiscence, wound infection, damage to blood vessels, or other cardiovascular system compromises
- Development of respiratory problems

4.3. WARNINGS AND PRECAUTIONS

Please, be sure to discuss with your surgeon the potential physical and psychological restrictions, the consequences of implanting the device and the surgical risks and possible side effects. If you have symptoms that appear abnormal, return for further consultation with your surgeon.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

The IntraSPINE® prosthesis, trial prostheses and ancillaries must only be used by a qualified surgeon who is trained in spinal surgery and to the surgical technique of IntraSPINE®. The information contained in this notice is necessary but not sufficient to control the surgical technique.

The prescription of the device is decided on by the surgeon, the only person qualified to do so

MRI SAFETY

The implants are composed of non-ferromagnetic materials and present a geometry not likely to generate induced current. Moreover, as they are fixed to tissues, they are unlikely to be mobilised. At first sight they can be considered compatible with an MRI scan. Their safety, in particular in terms of heating and migration of the implant, has been evaluated through bibliographic data by comparison with data available on devices with similar composition, shape and use. This evaluation concluded a conditional compatibility for MRI scan up to 1.5 Tesla.

As a precautionary measure, it is recommended MRI scans be avoided within 48 hours of the implant placement, and to inform the person in charge of the scan of the recent implant placement, if such examination is essential.

It should be noted that devices presenting a high contrast with the biological environment can generate “artifacts” that must be taken into account for the perfect execution and interpretation of imaging exams. For this purpose, the patient who has this implant must be advised to warn the health professionals concerned (radiologists and radiology operators) in as far as possible about the presence of this implant before these exams.

An implant card for the patient is provided by the healthcare professional at the clinic. This implant card provides information, for the patient, to identify the device and traceability elements as well as the name, address and website of the manufacturer. Please, scan the implant card immediately upon receipt to keep track of it in case of loss.

The summary of the safety and performance characteristics of the device and the instructions for use can be found on the COUSIN BIOTECH website.

The instruction for use is in electronic format and can be downloaded from the COUSIN BIOTECH website (www.cousin-biotech.com/ifu).

4.4. SUMMARY OF ANY FIELD SAFETY CORRECTIVE ACTION (FSCA INCLUDING FSN) IF APPLICABLE

There has been no Field Safety Corrective Action (FSCA) or Field Safety Notice (FSN) since the CE marked in 2007.

5. SUMMARY OF CLINICAL EVALUATION AND ON POST-MARKET CLINICAL FOLLOW-UP

5.1. CLINICAL BACKGROUND OF THE DEVICE

Sterile Devices for Interspinous Space with Laminal Support was already CE-marked and available on the European market since March 2007 (Directive 93/42/CEE).

The CE certificate was obtained for the RCBINTRAXX and BIOINTRAXX references, in accordance with European Regulation 2017/745, on 2025-05-09.

As part of the new Medical Device Regulation certification of the products, no novel feature has been introduced. The only change that took place in December 2025 concerned the removal of references created under Regulation 2017/745 and the addition of references previously used under Directive 93/42/CEE. These references are strictly identical.

The specifications of the final product remain unchanged. No change regarding the clinical performances and benefits.

5.2. CLINICAL EVIDENCE FOR THE CE MARKING

The clinical evidence is based on the following sources:

SCIENTIFIC LITERATURE

Clinical studies have been published in the scientific literature. They include clinical results on 662 patients who received the device. These clinical data have been used to demonstrate the clinical performance and benefits of IntraSPINE® and the acceptability of the benefits risk-ratio.

CLINICAL STUDIES

COUSIN BIOTECH has been performed two clinical studies regarding post-market clinical follow-up.

- 1) The first study is a 3 year prospective study of surgical outcomes for the IntraSPINE® device in patients treated for degenerative spinal pathologies with lumbar disc disease.

The objective was to evaluate the results of patients with symptomatic Lumbar degenerative disc disease treated with an IntraSPINE® device and followed up over a 3-year period. 231 patients were recruited and 180 completed the study.

In summary, the findings of this study are promising and demonstrate that successful patient outcomes can be achieved using the IntraSPINE® device for a range of degenerative diseases, with improvements in both back/leg pain and disability, sustained at 3 years. Implantation of the IntraSPINE® device was straightforward and there were no device-specific complications. The findings of this study coupled with other published research results suggest that IntraSPINE® may confer additional benefits in terms of back pain reduction and disability outcomes when used in conjunction with decompression or microdiscectomy in comparison to stand-alone decompressive procedures.

- 2) The second study calls “Efficacy and safety of the inter-laminar device IntraSPINE® associated with discectomy in the treatment of lumbar disc herniation”.

The primary objective is to assess at month 12 post-operatively the effect of IntraSPINE® device associated with discectomy on the degree of functional disability associated with lumbar pain following surgery for the treatment of lumbar disc herniation. The first inclusion of patient was on January 2019 and the last inclusion of patient is planned on June 2023.

198 patients will be included over 24 months in both arm. For the moment, 92 patients have been included in IntraSPINE® arm.

The clinical study is ongoing. COUSIN BIOTECH follows the advancement of the study and writes interim reports. For the moment, the analysis shows persistent benefit from surgery using IntraSPINE® between inclusion visit and 24 months follow up. The follow-up results are statistically significantly better than the preoperative status in every measurement.

The patients’ pain was improved, with greater gains demonstrated for relieving leg pain. Oswestry scores dropped from a preoperative mean of 55.9 % to a mean of 2.4 % after 24 months follow up.

Regarding complication, one patient experienced device migration, one patient had a removal of IntraSPINE® for suspicion of re herniation and one patient had a removal of IntraSPINE® for re herniation. Two patients had a replacement of IntraSPINE® device, one of them was because of right lumbosacral relapse and walking disorders.

The 24-months follow-up of patients with lumbar disc herniation after implantation of IntraSPINE® showed very good clinical and functional results. Like-wise, the consumption of analgesics by patients has sharply decreased.

These preliminary results show good clinical performance, and the benefit-risk ratio seems favourable. However, it is necessary to wait until the end of the clinical trial to conclude on the main objective (Oswestry Disability Index).

CLINICAL SURVEYS

On 23 November 2021, COUSIN BIOTECH sent surveys to surgeons who are users of IntraSPINE® implant for many years (France and worldwide), to know if they have already implanted the device in L1/L2 and/or L2/L3 levels, and to collect clinical data for these cases.

The selection of surgeons was made on surgeons with a high volume of IntraSPINE® use to obtain statistically representative data with usage on all possible implant levels.

In summary, IntraSPINE® device has already been implanted in four (4) cases in L1/L2 level, and in twenty (20) cases in L2/L3 level. No complication has been reported.

POST MARKET SURVEILLANCE OF THE DEVICE

Surveillance of the device is also an important source of clinical information.

Since first use in 2007, COUSIN BIOTECH has received 73 customer complaints for 82 000 devices sold. The total number of customer complaints is low compared to number of products sold: 0,088%.

In addition, 8 isolated incidents cases have been reported to competent authorities since its launch in 2007.

Supporting by the clinical evidence issued of the sources described above, the clinical performances of IntraSPINE® are the following:

- General performances outcome:
 - Treat chronic low back pain
- Specific performance outcomes
 - Full Motion preservation
 - Elastic support device with a cushioning effect
 - Ligaments assist the posterior ligament during flexion
 - The thin ligament stabilizes the interlaminar device on its site
 - Respect of Instantaneous Axis of Rotation of the segment
 - Wings: anti migration system
 - Allows Minimal invasive surgery
 - Fast and simple procedure (distraction and optimal positioning of the device)

Regarding the intended benefits for the patient, they are the following:

- Positive impact on clinical outcome: Less pain and less disability
- Patient's quality of life: Improvement of the quality of life

Consequently, there is another benefit with public health impact as the percentage return to work is improved.

Regarding the clinical risks of devices, the data collected available are of sufficient amount and quality for the detection of undesirable side-effects and their frequency.

No unexpected undesirable events have been detected in the clinical data available. All events detected are within expected limits and consistent with state of the art.

It can be concluded that the benefit risk of the device is acceptable.

5.3. SAFETY

Clinical risks are within the rates expected with this type of surgery. This confirms acceptability of the device benefit/risk profile according to current knowledge in this medical field and available medical alternatives.

The manufacturer continuously collects clinical data and ensures yearly updates of this summary of safety and clinical performance. The next yearly update will include the results of

- The clinical study "Efficacy and safety of the inter-laminar device IntraSPINE® associated with discectomy in the treatment of lumbar disc herniation" with the patient
- The scientific literature review
- The post market surveillance of the device

6. POSSIBLE DIAGNOSTIC OR THERAPEUTIC ALTERNATIVES

When considering alternative treatments, it is recommended to contact your healthcare professional who can take into account your individual situation.

Nonsurgical approaches

Non-surgical options include drugs, physiotherapy, spinal injections, lifestyle modification, and multidisciplinary rehabilitation. It is generally admitted in the guidelines that these approaches should always be considered as first choice given the limited level of demonstration of the benefit of surgical technique on the long term. Conservative interventions also appear to reduce fear (especially fear of falls) among patients with low back pain.

Nevertheless, when all applicable non-surgical approaches have been tried and failed for at least several months, invasive approaches can be proposed, with a careful weighing of the benefit risk of the intervention for the patient.

Invasive / Surgical approaches

In the majority of cases, spinal disc herniation articular facets syndrome doesn't require surgery. Surgery may be useful in those with a herniated disc that is causing pain and neurological symptoms in the lower limbs. (NASS guidelines). If there is incontinence, weakness and genital numbness (Cauda Equina syndrome) it is considered a medical emergency requiring immediate attention and possibly surgical decompression. Several techniques are described in the literature:

- **Discectomy** (removal some or all of a damaged disc, which can be helpful if it's pressing on a nerve).
- **Fusion surgery** (in case of a severely damaged disc, it may be removed. To re-stabilize this part of the spine, the surrounding vertebrae are fused (permanently joined) together).
- **Artificial disc replacement** (The damaged disc is removed and replaced with an artificial, or prosthetic, disc.)

Dynamic Stabilization: the implanted device (dynamic rod, interspinous or interlaminar cushion) will control the range of motion and allow to restore the disc height, decompressing the nerve structure and support the disc by taking off a part of the load it's bearing.

REVISION HISTORY

SSCP revision number	Date issued	Change description	Revision validated by the Notified Body
Issue 1	03/12/2021	Creation of the document	<input type="checkbox"/> Yes Validation language : <input type="checkbox"/> No (only applicable for class IIa or some IIb implantable devices (MDR, Article 52 (4) 2 nd paragraph) for which the SSCP is not yet validated by the NB)
Issue 2	29/07/2022	Update according to KIWA review	<input type="checkbox"/> Yes Validation language : <input type="checkbox"/> No
Issue 3	30/01/2023	Update according to KIWA review (Section 8. REFERENCE TO ANY HARMONISED STANDARDS AND COMMON SPECIFICATIONS APPLIED")	<input type="checkbox"/> Yes Validation language : <input type="checkbox"/> No
Issue 4	26/05/2023	Update according to KIWA review	<input type="checkbox"/> Yes Validation language : <input type="checkbox"/> No
Issue 5	27/02/2024	Update of the document to add the new headquarter address of Cousin Biotech in the technical file.	<input type="checkbox"/> Yes Validation language : <input type="checkbox"/> No
Issue 6	03/12/2025	Inclusion of the former reference used under the Directive (MDD 93/42/EEC), following requests from distributors who continue to use the historical references and removal of the reference that had been created to replace it under the MDR. Update following the correction of a numerical error identified in the company prefix of the Basic UDI-DI. Update following the certification of CE marking in compliance with Regulation (EU) 2017/745 on medical devices (MDR). Update of the template revision	<input type="checkbox"/> Yes Validation language : <input type="checkbox"/> No

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Titel der Norm	Datum
MDCG 2020-10/2 Clinical Investigation Summary Safety Report Form v1.0	Oktober 2022
MDCG 2020-13 Clinical evaluation assessment report template	Juli 2020
MDCG 2020-8 Guidance on PMCF evaluation report template	April 2020
MDCG 2020-7 Guidance on PMCF plan template	April 2020
MDCG 2020-6 Guidance on sufficient clinical evidence for legacy devices	April 2020
MDCG Clinical Evidence needed for medical devices previously CE marked under Directives 93/42/EEC or 90/385/EEC. A guide for manufacturers and notified bodies	April 2020
MDCG 2019-9 Summary of safety and clinical performance	März 2022
MDCG Registration of legacy devices in EUDAMED	April 2019
Timeline for registration of device data elements in EUDAMED	April 2019
MDCG Guidance Document implant card on the application of Article 18 Regulation (EU) 2017/745 on medical devices	März 2020
MDCG The EMDN - The Nomenclature of use in EUDAMED	Januar 2020
MDCG The CND Nomenclature - Background and general principles	Januar 2020
Guidance on Basic UDI-DI and changes to UDI-DI	Juni 2020
MDCG guiding principles for issuing entities rules on basic UDI-DI	Juni 2020
Guidance on harmonised administrative practices and alternative technical solutions until EUDAMED is fully functional	Mai 2021

BIBLIOGRAPHIE

¹Guizzardi G. IntraSPINE, an interlaminar, not interspinous, posterior motion preservation device in Lumbar DDD: indications and clinical results (over 2 year follow-up). Revista Latino-Americana de Neurocirugia. 2011 Nov;18:13–20

²Guizzardi G, Morichi R. Clinical Results with IntraSPINE®. EC Orthopaedics. 2015;2:101–6.

³Gelder CL, Drozda M, Spink G. LUMBAR MICRODISCECTOMY WITH INTRASPINE® – A CASE SERIES. Coluna/Columna. 2020 Jul;19(3):201–4.

⁴Co-Minh D. Surgical outcomes of patients with degenerative lumbar disc disease post-IntraSPINE® device fixation: Three-year prospective study. Herausgeber: Auctores Publishing LLC. IJCCR. 2021 Aug 11;8(1):01-10.

⁵Bae J, Lee SM, Lee S-H, Shin S-H, Kim H-J, Kim KH. The Likelihood of Reaching Substantial Clinical Benefit After an Interlaminar Dynamic Spacer for Chronic Low Back Pain: A Clinical and Radiologic Analysis of a Prospective Cohort. World Neurosurgery. 2017 May;101:589–98.

⁶Corriero OV. Lumbar Herniated Disc Treated by Microdiscectomy Alone or Microdiscectomy Plus an Interlaminar Shock Absorbing Device: Retrospective Study with Minimum 3-Years Follow-Up. J Spine [Internet]. 2013 [zitiert 24. Juli 2019];03(02). Abrufbar unter: <http://www.omicsgroup.org/journals/lumbar-herniated-disc-treated-by-microdiscectomy-alone-or-microdiscectomy-plus-an-interlaminar-shock-absorbing-device-2165-7939-3-159.php?aid=23954>

⁷Galarza M, Gazzeri R, De la Rosa P, Martínez-Lage JF. Microdiscectomy with and without insertion of interspinous device for herniated disc at the L5–S1 level. Journal of Clinical Neuroscience. 2014 Nov;21(11):1934–9.

⁸Feng S, Fan Z, Li X, Lin J, Yang Y, Fei Q. Combination of Intraspine Interlaminar Device With Lumbar Discectomy in Patients With L5/S1 Disc Herniation [Internet]. In Review; 2020a Sep. Abrufbar unter: <https://www.researchsquare.com/article/rs-64740/v1>

⁹Bistazzoni S, Angelis MD, D'ercole M, Chiamonte C, Carotenuto A, Cardarelli G, et al. Evaluation of Effect of Posterior Dynamic Stabilization IntraSPINE System on Sagittal Spinal Balance using EOS® X-Ray Imaging System. J Neurol Neurophysiol [Internet]. 2017 [zitiert 24. Juli 2019];08(04). Abrufbar unter: <https://www.omicsonline.org/open-access/evaluation-of-effect-of-posterior-dynamic-stabilization-intraspin-systemon-sagittal-spinal-balance-using-eos-xray-imaging-system-2155-9562-1000439.php?aid=92605>

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² Guizzardi G, Morichi R. Clinical Results with IntraSPINE®. *EC Orthopaedics*. 2015;2:101–6.

³ Gelder CL, Drozda M, Spink G. LUMBAR MICRODISCECTOMY WITH INTRASPINE® – A CASE SERIES. *Coluna/Columna*. 2020 Jul;19(3):201–4.

⁴ Co-Minh D. Surgical outcomes of patients with degenerative lumbar disc disease post-IntraSPINE® device fixation: Three-year prospective study. *Auctores Publishing LLC, editor. IJCCR*. 2021 Aug 11;8(1):01–10.

⁵ Bae J, Lee SM, Lee S-H, Shin S-H, Kim H-J, Kim KH. The Likelihood of Reaching Substantial Clinical Benefit After an Interlaminar Dynamic Spacer for Chronic Low Back Pain: A Clinical and Radiologic Analysis of a Prospective Cohort. *World Neurosurgery*. 2017 May;101:589–98.

⁶ Corriero OV. Lumbar Herniated Disc Treated by Microdiscectomy Alone or Microdiscectomy Plus an Interlaminar Shock Absorbing Device: Retrospective Study with Minimum 3-Years Follow-Up. *J Spine [Internet]*. 2013 [cited 2019 Jul 24];03(02). Disponibile in: <http://www.omicsgroup.org/journals/lumbar-herniated-disc-treated-by-microdiscectomy-alone-or-microdiscectomy-plus-an-interlaminar-shock-absorbing-device-2165-7939-3-159.php?aid=23954>

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⁵ ae J, Lee SM, Lee S-H, Shin S-H, Kim H-J, Kim KH. The Likelihood of Reaching Substantial Clinical Benefit After an Interlaminar Dynamic Spacer for Chronic Low Back Pain: A Clinical and Radiologic Analysis of a Prospective Cohort. World Neurosurgery. 2017 Mayo;101:589-98.

⁶ Corriero OV. Lumbar Herniated Disc Treated by Microdiscectomy Alone or Microdiscectomy Plus an Interlaminar Shock Absorbing Device: Retrospective Study with Minimum 3-Years Follow-Up. J Spine [Internet]. 2013 [citado 2019 jul. 24];03(02). Disponible en: <http://www.omicsgroup.org/journals/lumbar-herniated-disc-treated-by-microdiscectomy-alone-or-microdiscectomy-plus-an-interlaminar-shock-absorbing-device-2165-7939-3-159.php?aid=23954>

⁷ Galarza M, Gazzeri R, De la Rosa P, Martínez-Lage JF. Microdiscectomy with and without insertion of interspinous device for herniated disc at the L5–S1 level. *Journal of Clinical Neuroscience*. 2014 nov.;21(11):1934-9.

⁸ Feng S, Fan Z, Li X, Lin J, Yang Y, Fei Q. Combination of Intraspine Interlaminar Device With Lumbar Discectomy in Patients With L5/S1 Disc Herniation [Internet]. En revisión; 2020a Sep. Disponible en: <https://www.researchsquare.com/article/rs-64740/v1>

⁹ Bistazzoni S, Angelis MD, D'ercole M, Chiaramonte C, Carotenuto A, Cardarelli G, et al. Evaluation of Effect of Posterior Dynamic Stabilization IntraSPINE System on Sagittal Spinal Balance using EOS® X-Ray Imaging System. *J Neurol Neurophysiol* [Internet]. 2017 [citado 2019 jul. 24];08(04). Disponible en: <https://www.omicsonline.org/open-access/evaluation-of-effect-of-posterior-dynamic-stabilization-intraspine-system-on-sagittal-spinal-balance-using-eos-xray-imaging-system-2155-9562-1000439.php?aid=92605>

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